Cercon® eye Scan-Spray

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DequDent
A Dentsply International Company

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Instructions for use (Revision: 02/07)

Cercon eye Scan-Spray is used to achieve optimum recordings for CAD/CAM restorations. The spray can be used lab-side (extraorally applied on the stone model). The Spray balances out the different optical properties of the stone model and thus permits optimum recording using a camera and/or scanner. With the atomizing nozzle, an optimum covering layer with a detailed representation of the edges is achieved easily and efficiently with only a short spray discharge.

Warnings:

Highly flammable; Do not breathe spray; Use only in well-ventilated areas.

Composition:

Pigment suspension in ethanol. The propellant is a fluorinated hydrocarbon.

Indication:

Extra-oral application (plaster die/plaster model) for lab-side use.

Application restrictions:

Application of the spray on unsuitable and dried surfaces.

General note:

Failure to observe the application restrictions or stipulated working steps may result in failure.

Side effects:

Side effects are not known to date. In individual cases, allergic reactions may occur. If a patient is suspected or known to be allergic to any of the ingredients. Scan spray should no longer be used.

Processing:

Preparation for spraying

1. Attach spray cannula to the can and turn the nozzle to the correct position.
2. The nozzle can be rotated 360° to achieve an optimum position to reach the areas to be spray coated.
3. The spray cannula can be slightly bent. However, never kink it at a right angle.
4. Clean the surface to be spray coated and dry thoroughly.

Spray procedure:

1. Carry out one short spray discharge per tooth.
2. The can must always be kept upright during spraying.
3. The spray head must be entirely pressed down during spraying to prevent premature clogging of the atomizing nozzle.
4. The distance from the nozzle to the surface to be coated should be approximately 4–6 cm.
5. Conduct recording using the camera or scanner as usual.

Cleaning of the surfaces:

Once the recording has been completed, the surface can be cleaned using water and steam jet (lab-side). To achieve thorough cleaning of the surfaces, the procedure may have to be repeated.

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