Cercon[®] mill

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Great care was taken in selecting the texts and illustrations used in this Instructions for Use. Nevertheless, this Instructions for Use may still contain typing errors or other mistaken data. Please note that DeguDent GmbH cannot be held liable for these errors.

1 Name of Each Components 21

2 Specifications 22

3 Installation Procedures of Each Component. 22

3.1 Installation of Plumbing Hose 22

3.2 Installation of Foot Pedal 22

3.3 Installation of Handpiece 23

3.4 Installation and Removal of Water Supply Bottle 23

3.5 Installation and Removal of the Water Supply Bottle Cap. 24

3.6 Installation and Removal of Water Filter Set (Use of City Water) 24

4 Operational Procedures of Control Unit. 25

4.1 Set the Drive Air Pressure 25

4.2 Set the Water Supply 25

4.3 Operation 25

4.4 Adjustment of the Amount of Water Supplied (when water supply is used) 26

4.5 Adjustment of chip air (when water supply is used) 26

4.6 For Finishing Work. 26

5 Operational Procedures of Handpiece. 26

6 Replacement Procedures of Cartridge 27

7 Replacement or Cleaning Procedures of Chuck 27

7.1 Removal of Chuck. 27

7.2 Installation of a New Chuck. 28

8 Maintenance of Unit 29

8.1 Draining the filter 29

8.2 Changing O-ring (Use of Water Supply Bottle) 29

8.3 Changing Water Filter (Use of City Water) 29

9 Declaration of conformity for equipment. 30


Caution

- Cercon® mill is a handpiece used in dental laboratory work for cutting while preventing heat when performing fine work on porcelain teeth, porcelain and ceramics. Do not use for other applications.
- When cutting, wear safety glasses, and use a grinding box and vacuum for safety.
- Do not modify or disassemble unnecessarily. Otherwise, performance and safety may be seriously affected. Please ask your Cercon® mill dealer whenever repairs are required.
- Do not drop the handpiece and unit. Make sure to put the unit on a horizontal and stable surface.
- Observe the permissible speeds the manufacturer or dealer designates for the bur. Otherwise, you may be injured by scattering and breaking up of the bur.
- Do not use burs which are oscillating or bent or damaged and of which shank portions are worn. Otherwise, you may be injured by scattering and breaking up of the bur.
- Do not turn the chuck release ring during rotation of the handpiece. Otherwise, the handpiece will be broken.
- Never supply lubricant to the handpiece. It uses a grease filled bearing and additional lubrication may cause failures.
- Pay careful attention to rattling, vibrations, sounds and temperature (heating), and inspect by rotating in advance before starting use. Immediately ask your dealer for repair work if any irregularities are sensed.
- The proper air pressure is 0.25~0.35 MPa. Pressure exceeding the value may lead to premature failure of the bearing because of excessive increases in speeds.
- Extract water if water drips or contaminants are found on the filter of the unit.
- Always clean the shank of the bur you are going to mount. Contaminant deposits in the chuck may lead to oscillating of the bur and breakage of the chuck.
- Install the cutting bur or test bur even when not in use.
- Operational control, maintenance and inspection is the responsibility of each user.

Name of Each Components 1

- 1 Cercon® mill Control Unit
- 2 Filter Inspection Window
- 3 Pressure Gauge
- 4 Regulator Knob
- 5 Chip Air Adjustment Knob
- 6 Water Adjustment Knob
- 7 Water ON/OFF Switch
- 8 Water Supply Bottle
- 9 Handpiece Connector
- 10 Water Filter Set
- 11 Foot Pedal
- 12 Plumbing Hose
- 13 Cercon® mill Handpiece
- 14 Coupling Joint
- 15 Handpiece Hose
- 16 Bottle Attaching/Detaching Switch
- 17 Water Mode Select Switch
- 18 Foot Pedal Connector
- 19 Hose Stopper
- 20 Water Connector
- 21 Plumbing Connector
- 22 Bur-inserting Wrench
- 23 Rotor Shaft Nut Wrench
- 24 Air Supply Connector
- 25 Chuck Release Ring
- 26 Cartridge
- 27 Cartridge retainer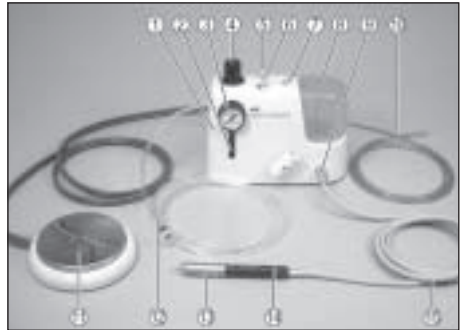

2 Specifications

Speed:

320.000 min⁻¹

Handpiece Dimensions:

Ø 16,6 mm x H 136,6 mm, 0.65 in/5.37 in

Control Unit Dimensions:

W 214 mm x D 110 mm x H 175 mm
W 8.42 in/D 4.33 in/H 6.89 in

Proper Air Supply Pressure:

0,25~0,35 MPa

Handpiece Weight:

104 g (including coupling joint), 3.67 oz

Control Unit Weight:

1,3 kg, 2.86 lb

3 Installation Procedures of Each Component

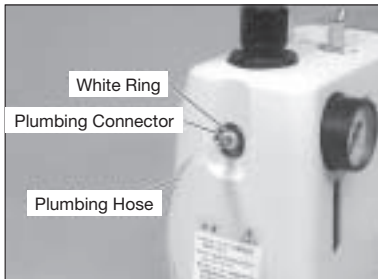
Fig. 1

3.1 Installation of Plumbing Hose

Push the plumbing hose into the plumbing connector located at the regulator on the left-hand side of the control unit until it is securely set to make connection as shown in Fig. 1. Check if the hose is securely installed by slightly tugging it after connection. Connect the other end of the plumbing hose to the air line. At this time, use the attached connector if necessary.

- Push the plumbing hose into the connector until it is securely set. Otherwise, air may leak.
- Pushing the white ring, on the plumbing connector, gently remove the tube.

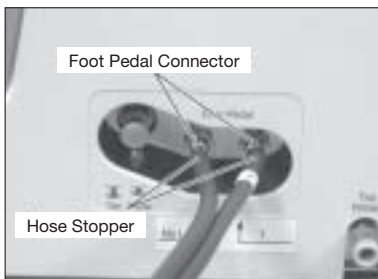
Fig. 2

3.2 Installation of Foot Pedal

Loosen and remove the hose stoppers (two) from the foot pedal connectors on the back of the control unit, and insert them into the foot pedal hoses. At this time, insert the hose stoppers so that their screws come outside (control unit side). Insert the „1“-marked end of each hose according to the instructions on the label. After insertion of both hoses, securely tighten the hose stoppers (Fig. 2).

3.3 Installation of Handpiece

Install the handpiece hose to the handpiece connector on the front face of the unit (Fig. 3). Install the handpiece hose to the coupling joint (Fig. 4). And then, install the handpiece by pulling the ring of the coupling joint (Fig. 5).

Fig. 3

Fig. 4

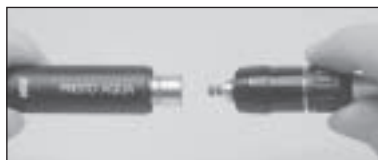
Fig. 5

3.4 Installation and Removal of Water Supply Bottle

Installation: Push in the bottle attaching/detaching switch on the back of the control unit until it is locked (Fig. 6). Insert the water supply bottle from above, and push it in until the bottle attaching/detaching switch returns properly. Check to see if the water supply bottle is securely installed by pulling it upward.

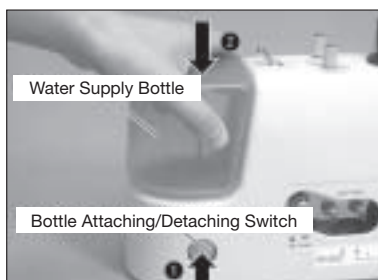
Fig. 6

- Check to see if the bottle attaching/detaching switch has returned properly and the water supply bottle is securely installed. Otherwise, water and air may leak, and the water supply bottle may pop out.

3 Installation Procedures of Each Component

Fig. 7

3.4 Installation and Removal of Water Supply Bottle

Removal: Holding the water supply bottle, push in the bottle attaching/detaching switch until it is locked, and as it is pushed in, while holding the control unit, pull out the water supply bottle upward (Fig. 7).

- After use, if water is low in the water supply bottle, the water supply bottle may pop out; therefore, the bottle attaching/detaching switch should be pressed as the water supply bottle is held.

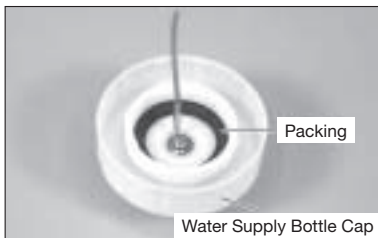
Fig. 8

3.5 Installation and Removal of the Water Supply Bottle Cap

Installation: Check the packing inside the water supply bottle cap for misalignment or detachment, and then close the water supply bottle cap (Fig. 8).

Removal: Before opening the water supply bottle cap, be sure to press the bleeder button to release the pressure in the water supply bottle (Fig. 9). Check that the pressure is completely released, and then open the water supply bottle cap.

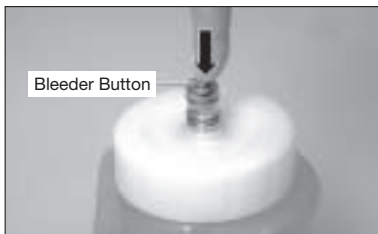
Fig. 9

3.6 Installation and Removal of Water Filter Set (Use of City Water)

Push the hose of the water filter set (the side of the water filter with a case) into the tap water connector on the back of the control unit until it is securely set to make connection as shown in Fig. 10. Check if the hose is securely installed by slightly tugging it after connection. Connect the other end of the hose to the tap water line.

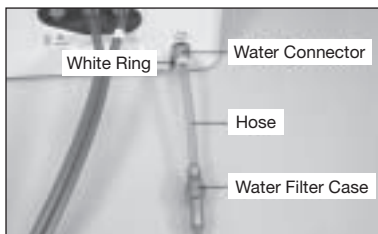
Fig. 10

- Push the hose of the water filter set into the tap water connector until it is securely set. Otherwise, water may leak. Pushing the white ring, on the water connector, gently remove the tube.

4.1 Set the Drive Air Pressure

Supply air, and adjust to 0.3 MPa by pulling the regulator knob upward and turning. When adjustment is made, push the knob down to lock.

4.2 Set the Water Supply

For use with water supplied, set the water ON/OFF switch to the ON side. **Use of Water Supply Bottle:** Push the water mode select switch until it stops (Fig.11). **Use of City Water:** Pull the water mode select switch until it stops (Fig. 11). Open the main cock of the tap water.

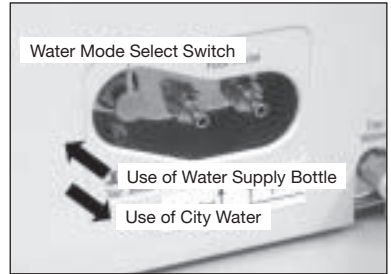
Fig. 11

- Properly set the water mode select switch because if it is stopped at an intermediate position, air is released structurally.

4.3 Operation

Stepping on the foot pedal starts rotation. If the water ON/OFF switch is set to the ON side, water supply is also started.

4 Operational Procedures of Control Unit

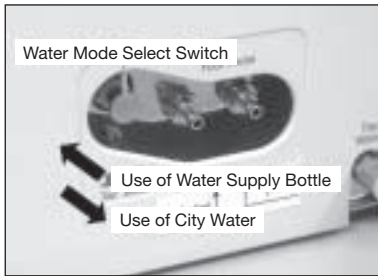
Fig. 11

4.4 Adjustment of the Amount of Water Supplied (when water supply is used)

Check the amount of water supplied, and adjust the amount of water supplied with the water adjustment knob.

- When tap water is used, the water pressure may differ according to the installation location, resulting in changes in the amount of water supplied; therefore, adjustment should be made in each case.

4.5 Adjustment of chip air (when water supply is used)

Check the spraying condition, and adjust the amount of chip air with the chip air adjustment knob. Increasing the chip air results in a mist state, and decreasing it results in a straight-line state.

4.6 For Finishing Work

After completion of operation, close the main air cock.
Use of City Water: Close the main cock of the tap water.

5 Operational Procedures of Handpiece

Fig. 12

1. As the chuck of the Cercon[®] mill handpiece is a pushin type, set FG instruments of $\varnothing 1.6$ mm and push in as shown in Fig. 12. Push in the instrument of which the end is tapered by using the designated bur-inserting wrench as shown in Fig. 13.

Fig. 13

2. When removing the instrument, it can be removed by turning the chuck release ring in the direction of the arrow in Fig. 14.

Fig. 14

Replacement Procedures of Cartridge 6

1. Loosen the cartridge retainer by turning it, in the direction shown in Fig. 15.

2. Hold the instrument and pull out the cartridge toward the front (Fig. 16). At this time, the O-ring placed on the rear bearing may remain in the body. Make sure to remove the O-ring. Before inserting a new cartridge, clean the head interior.

3. When mounting a new cartridge, align the position of the pipe for water and chips, and the positioning pin with the hole of the nose hood of the cartridge and push in straight (Fig. 17).

4. Finally, tighten the cartridge retainer securely.

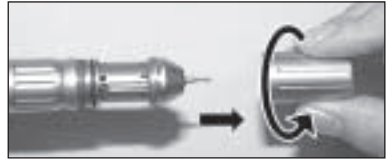
Fig. 15

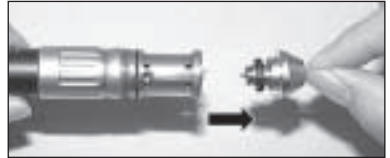
Fig. 16

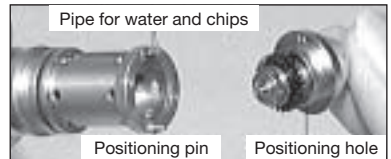
Fig. 17

Replacement or Cleaning Procedures of Chuck 7

7.1 Removal of Chuck

1. Remove the cartridge with a instrument as it is being installed. (6. Replacement Procedures of Cartridge)

2. Hold the rotor with two fingers as shown in Fig. 18. Mount the wrench provided on the rotor nut located on the top of rotor shaft and turn counterclockwise. (Either side of wrench can be used.)

Fig. 18

7 Replacement or Cleaning Procedures of Chuck

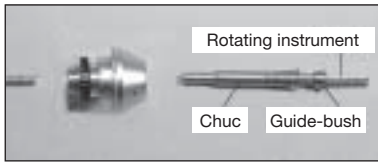
Fig. 19

3. When the rotor nut is loose, push out the chuck carefully with the shank of the rotating instrument from the back of the cartridge, the Guide-bush and the chuck will come out (Fig. 19).

- As the Guide-bush is a very small part, be careful not to lose it.

When cleaning the chuck, clean it sufficiently with ultrasonic cleaner etc. Blow oil spray to the inside of the rotor shaft and clean thoroughly.

- Remove the chuck and clean once a week.

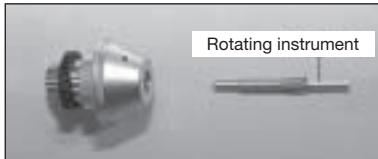
Fig. 20

7.2 Installation of a New Chuck

1. Apply oil thinly on the surface of the new chuck or the chuck after cleaning, insert the rotating instrument and push into the rotor shaft (Fig. 20).

2. Set the longer one of the guides of the guide bush on the end of the rotor shaft first along the rotating instrument (Fig. 21).

3. Finally insert the rotor shaft nut along the rotating instrument, and firmly tighten it with a wrench while holding the rotor (Fig. 22).

Fig. 21

Fig. 22

8.1 Draining the filter

Check the filter bowl from the filter inspection window, and if there is an accumulation of water, insert fingers from the bottom, and turn the drain cock in the direction of the arrow to drain water from the filter (Fig. 23). After draining, securely tighten the drain cock by turning it in the reverse direction.

Fig. 23

8.2 Changing O-ring (Use of Water Supply Bottle)

Remove the two O-rings at the bottle joint with a pointed tool, and mount a new nose into the grooves (Fig. 24).

Optional O-ring: Order No.Y900-327

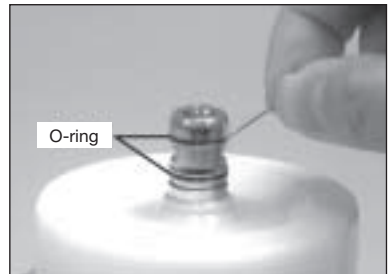
Fig. 24

8.3 Changing Water Filter (Use of City Water)

1. Close the water valve of to connect the Cercon® mill. Mount two (5 x 8) spanner wrenches as shown in Fig. 25, and turn in the direction shown. When the water tube becomes twisted as a result of this, relieve the twisting by turning it, as its end on the control unit side is free to turn.

2. When the water filter case is separated, the water filter can be removed as shown in Fig. 26. Replace with a new one and reassemble the filter in the reverse order (Fig. 26).

Water Filter: Order No. U387-042

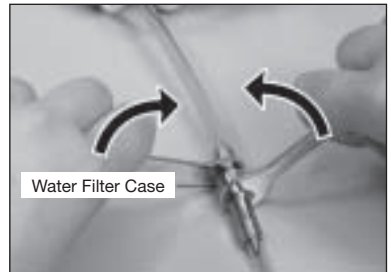
Fig. 25

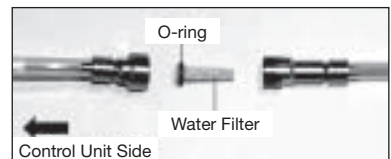
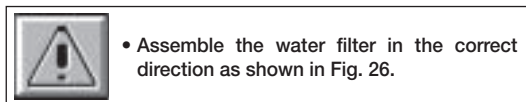
Fig. 26

9 Declaration of conformity for equipment

Declaration of conformity for equipment (Formular Q-04-R-007)

Last revision: 30.05.2007

For

DeguDent GmbH
Rodenbacher Chaussee 4
D-63457 Hanau

we declare that

Document number: Q-04-KE-14-3004

Revision index: 0

Cercon® mill

Lube-Free Lab Air Turbine with Water Coolant
Product number: 5355700001

complies with the requirements of the following guideline(s)


1. EC machinery directive 98/37/EG

The following harmonized standards have been used:

DIN EN 1050: 1996

Hanau, 30.05.2007
Dr. Udo Schusser
Vice President Research
and Development

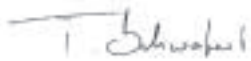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