CAD/CAM

cercon[®]

Cercon[®] heat plus P8

Gebrauchsanweisung/Instructions for Use/Mode d'emploi/
Istruzioni per l'uso/Instrucciones de uso

DeguDent
A Dentsply Company

D	Gebrauchsanweisung	2
GB	Instructions for Use	24
F	Mode d'emploi	46
I	Istruzioni per l'uso	68
E	Instrucciones de uso	90

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The text and illustrations of these Instructions for Use have been compiled with the utmost diligence. Nevertheless, the presence of typographic errors or incorrect data cannot be excluded. Please note that DeguDent GmbH will not be responsible for such errors.

0	Content	25
1	General information	27
1.1	On these Instructions for Use	27
1.2	Layout elements used in these Instructions for Use	27
1.3	Device type and year of manufacture.....	27
1.4	Manufacturer and service addresses	27
1.5	Intellectual property rights	28
1.6	Purpose.....	28
1.7	Contraindications.....	28
1.8	Warranty and liability	28
2	Warning and safety messages.....	29
2.1	Moving and unpacking safety notes	29
2.2	Deployment and operation safety notes.....	29
2.3	Maintenance and troubleshooting safety notes	30
2.4	Emergency shutdown.....	30
2.5	Safe handling of ceramic fibres	30
2.6	Symbols on product labels	31
2.7	Thermal safety notes	31
3	Technical description.....	32
3.1	Base unit	32
3.2	Controls and connectors	33
3.3	Scope of delivery	33
3.4	Technical specifications and operating environment.....	33
4	Setting up the furnace.....	34
4.1	Moving and unpacking	34
4.2	Setting up and connecting.....	35
5	Controls.....	36
5.1	Control panel with membrane keyboard and display.....	36
5.2	General information on program execution	37
5.3	DISPLAY key, INFO key	38
6	Operation	39
6.1	Turning on the unit and preparing the sintering trays.....	39
6.2	Program selection and program start.....	39
6.3	Canceling program execution	40
6.4	Program execution	40
6.5	Program termination: Removing sintering trays.....	41

Contents

7	Cleaning	41
8	Error messages and troubleshooting	42
8.1	Problems	43
8.2	Error messages.....	44
8.3	Cracks in the insulation or retort tube	44
8.4	Lifting table, emergency lowering.....	45
8.5	Power failure	45
8.6	Returning the unit	45
9	Disposal	46
10	Declaration of Conformity	47

Dear customer:

Thank you for your confidence in Cercon® heat plus P8. This sintering oven is a safe high-quality device featuring numerous reliable and proven functions. Specifically, it is characterized by:

- Easy operation
- Minimal training requirements
- High-quality craftsmanship
- High performance and efficiency
- Low power consumption
- Minimal noise
- Easy deployment
- Easy maintenance

We are confident you will appreciate your results with the Cercon® heat plus P8.

1.1 On these Instructions for Use

Compliance with these Instructions for Use is a prerequisite for the successful and safe operation of the Cercon® heat plus P8. These Instructions for Use contain important information for operating this unit in a safe, appropriate and economic manner. Compliance with these Instructions for Use also helps avoid hazards, limits repair costs and downtime and increase the reliability and prolong the life of the Cercon® heat plus P8.

These Instructions for Use must be continuously present near the unit and must be read by all persons working with the Cercon® heat plus P8.

DeguDent GmbH will not be responsible for any damage caused by improper use or operation of the Cercon® heat plus P8 and/or non-compliance with these Instructions for Use.

1.2 Layout elements used in these Instructions for Use

Safety notes related to personal injury, accidents or property damage:
The unit must be opened only by authorized DeguDent GmbH service personnel!

Step-by-step instructions

1. Remove...
2. Locate...

Designations of user interface elements:

INFO key

Emphasis:

The front of the unit includes the display (1) with its...

Additional notes and hints:

Note: You may also use existing programs to...

1.3 Device type and year of manufacture

Cercon® heat plus P8

Year of manufacture: 2008

1.4 Manufacturer and service addresses

DeguDent GmbH

Rodenbacher Chaussee 4

63457 Hanau

Germany

Phone: + 49 6181 59 8663

Fax: + 49 6181 59 5658

www.degudent.com

For maintenance and repairs, please contact the DeguDent customer service. If you have any question, problems, or suggestions, please do not hesitate to contact us (see our contact parameters above). When calling us, please have your **ID/rating plate** information handy.

1 General information

1.5 Intellectual property rights

© 2008 DeguDent GmbH

All rights pertaining to these Instructions for Use, including but not limited to photocopying and dissemination as well as translation, are reserved for DeguDent GmbH. Specifically, these Instructions for Use must not, whether in whole or in part, be reproduced in any form (print, photocopy, microfilm or other technologies) and/or stored, processed, copied or disseminated using electronic data storage and retrieval systems without prior written approval by DeguDent GmbH.

Violations of these rights will be prosecuted and will make the guilty party liable for damages. All intellectual property, trademark, service mark and other rights are reserved by DeguDent GmbH.

1.6 Purpose

The Cercon® heat plus P8 is a device for **sintering Cercon® Zirkonoxid frameworks** as per the instructions in Section 3.

Any use outside or beyond those directions or specifications does not constitute proper use. Any damage resulting from such use will be the sole responsibility of the user of the Cercon® heat plus P8. The same is true of any unauthorized changes to the unit. Proper use presupposes compliance with all instructions on:

- Safety
- Operation
- Care, maintenance and troubleshooting

described in these Instructions for Use. The unit is intended solely for dental laboratory use. Any use in other locations or for other purposes requires prior written approval of DeguDent GmbH.

1.7 Contraindications

The Cercon® heat plus P8 is intended for **professional dental use only**. The Cercon® heat plus P8 is intended for professional dental use only. The use of Cercon® heat plus P8 for other activities other than dental use is not permitted. The Cercon® heat plus P8 may not be operated in areas with a combustible atmosphere.

1.8 Warranty and liability

This device is subject to the conditions of the **DeguDent Limited Warranty**. In addition, the following conditions apply:

DeguDent will not honor warranty claims or accept liability for any personal injury or property damage attributable to one or more of the following causes:

- Any person in charge of the operation, deployment, installation, maintenance, or repair of this unit must have read and understood these Instructions for Use. There will be no liability for any damage or operational irregularities arising from non-observation of these Instructions for Use.
- Inappropriate use of the device.
- Improper or unauthorized deployment, installation, commissioning, operation, or maintenance of the unit.
- Use of the unit in the presence of defective safety installations or inappropriately installed or non-functional safety or protection installations.
- Non-compliance with these Instructions for Use with regard to shipping, storage, deployment, installation, commissioning, operation, maintenance, or preparation.
- Unauthorized modifications to the device.
- Unauthorized modification of operation parameters, unauthorized modification of other parameters and settings, or unauthorized modification of programs.
- DeguDent original parts and accessories are designed specifically for DeguDent ovens. Use only original DeguDent replacement parts. The use of other parts voids the warranty. DeguDent will not accept liability for any damage caused by the use of non-original parts.
- Damage caused by catastrophic events, external interaction, or force majeure.

All DeguDent GmbH devices are designed and produced using **state-of-the-art technology** and following all recognize safety rules.

Nevertheless, using these devices may result in **hazards to operators or third parties or damage** to the Cercon® heat plus P8 or other property, e.g. if:

- The unit is operated by untrained or not properly instructed personnel.
- The unit is not employed for its appropriate uses.
- The unit has not been properly operated or maintained.

Use only **qualified and trained personnel** to perform to tasks described in these Instructions for Use. Observe any applicable legal age restrictions!

Personnel to be trained or guided, including vocational trainees, must work with the Cercon® heat plus P8 **only if continuously supervised** by an experienced operator!

2.1 Moving and unpacking safety notes

Please observe the following safety instructions, especially when moving and deploying the unit:

- The carton containing the unit and the unit itself are very heavy (**75 kg**). Use appropriate load lifting tackle.
- The Cercon® heat plus P8 must be lifted and carried by at least **two persons**. Use suitable **carrying straps** for greater distances.
- Do not lift the unit by the **lifting table** when unpacking to avoid damage to the guide rails and spindle drive.
- The unit is shipped in a **cardboard** case and is protected from shock and damage by appropriate **interior packaging material**. Please retain its packaging material and **reuse** when repackaging.

2.2 Deployment and operation safety notes

To ensure the **safe deployment** and **safe operation** of the unit, read and follow the following instructions:

- Before turning on the Cercon® heat plus P8 for the first time, **leave the unit to rest** at the intended place of employment for at least **24 hours** to allow any moisture to evaporate from the electrical system.
- Deploy the Cercon® heat plus P8 in a **dry room** (IP20) and protect the unit from moisture and steam.
- Use the unit only in **well-ventilated rooms**. If necessary, install an appropriate vacuum system, which must be operative when the unit is in use.
- The table on which the unit is deployed must be **capable** of safely sustaining the unit's weight of **75 kg**.
- The surface of the table as well as the floor must be made of **non-flammable** materials.
- Make sure to maintain a **safety distance** of **50 cm/19.69 in** on all sides.
- Make sure that the bottom of the sintering chamber does not have to bear loads of more than **2 kg**.
- Before connecting the power cord, make sure that the power switch is in the **OFF** position.
- Ensure that the unit is connected only to the **appropriate voltage** (see Technical specifications and operating environment).
- The Cercon® heat plus P8 must be the **only device** on its circuit.
- The wall outlet must be **grounded**, located **near** the **unit**, and **easily accessible**.
- When using **extension cords**, make sure that they are **fully grounded** and that all cable cross-sections meet the pertinent specifications.
- Immediately replace any **damaged** power or extension **cords**.
- Make sure to operate the Cercon® heat plus P8 only when the unit is in good technical shape and fully functional.
- All **safety messages** and **warning labels** on the device must be legible. Damaged or illegible labels must be replaced.
- Follow the **safety instructions** for Cercon® Zirkonoxid at all times.

2 Warning and safety messages

- **Burn hazard!** The areas in and around the sintering chamber may be hot, especially on top (> 65°C). **Wear protective gloves.**
- **Burn hazard!** Sintering trays may be hot. **Wear protective gloves.**
- **Fire hazard!** Never place any objects on the Cercon® heat plus P8, not even temporarily.
- Do not reach into the sintering chamber. **Burn hazard!**
- Do not reach into the sintering chamber to prevent damage to the retort tube or heating element.
- Crushing hazard when the lifting table closes! **Keep away** from all movable parts.
- Always place the sintering trays in the middle of the insulated floor of the sintering chamber. Observe the **MAX** mark when stacking trays to avoid damage to the installation, the retort tube, the heating elements, or the thermal element.
- Use only Cercon® heat plus sintering trays in the furnace. Cercon heat sintering trays may not be used. Warning: Improper tray use may result in damage to the heating elements!
- Opening the unit, is prohibited. **Electric shock hazard!**
- If the Cercon® heat plus P8 is not to be used for an extended period, disconnect the power cord.
- Pull the power supply plug before use the emergency lowering!

2.3 Maintenance and troubleshooting safety notes

Observe the following rules during **maintenance, repairs, or troubleshooting** tasks:

- All repairs must be performed by authorized DeguDent GmbH personnel only.
- Only qualified electricians are allowed to open the unit and to work on electrical components.
- Disconnect the power cord before opening the unit.
- Do not insert any objects into the ventilation slits. Protect from moisture.
- Power cords may only replaced by approved power cords with an identical rating.
- Use only original replacement parts.
- The heating elements of Cercon® heat plus P8 are to be replaced with daily usage after 12 months.

2.4 Emergency shutdown

An **emergency shutdown** is performed by **disconnecting the power cord**. During operation of the unit, the power cord must be accessible at all times to allow disconnecting the unit in an emergency.

2.5 Safe handling of ceramic fibers

The **heat insulation** of the sintering chamber consists of **ceramic fibers**. Extended use or higher temperatures will result in the formation of substances that, when released into the air, may create a **dust hazard**, causing **irritation to the skin or eyes** or the **respiratory tract**. The insulation material used for this furnace contains ceramic fibres. In the European Union, ceramic fibres have been rated as follows, pursuant to EC Directive 97/69/EC dated 5 December 1997:

CARC.CAT 2; R49; Xi R 38.

Never open and expose the heat insulation of the sintering chamber. Any repairs in this area must be performed by authorized DeguDent GmbH personnel only.

2.6 Symbols on product labels

- The **Hot Surface warning** decal alerts the operator to hot temperatures on Cercon® heat plus P8 surfaces that may prevail near the sintering chamber, especially in the case of higher sintering temperatures.
- High temperatures may be encountered especially at the top of the sintering chamber (**> 65 °C**) and on the sintering trays (**200 °C**)!
- The **Dangerous Voltage warning** decal on the device housing has the following meaning:
 - **Hazardous voltage** inside the unit.
 - **All repairs** must be performed by authorized DeguDent GmbH personnel only.
 - Only **by authorized DeguDent GmbH personell** are allowed to open the unit and to work on electrical components.
 - **Disconnect** the power cord before opening the unit.
 - **Do not place** any objects into the ventilation slits. Protect from moisture.
 - Use only accessories approved for use with Cercon® heat plus.
 - Use only **original** replacement parts.
 - See Instructions for use before usage of the unit.

2.7 Thermal safety notes

The unit is a **Thermal Protection Class 0** device pursuant to EN 60519-2 (without safety control).

The electrical circuitry of the unit is designed for operation at ambient air temperatures of between **+ 0 °C till + 40 °C**.

Humidity: 5 – 90% non-condensing.

In the presence of higher temperatures, **electrical** switch-gear **cooling** must be used. In the presence of high levels of relative humidity and very low temperatures, **heaters** must be used.


Make sure that all **covers are in place** and **properly fastened** at all times.

Storage temperature: -40 °C till + 65 °C.

Ambient temperature unit: 0 °C - + 40 °C.

Otherwise you may reckon the life time of the electrical endurance.

3 Technical description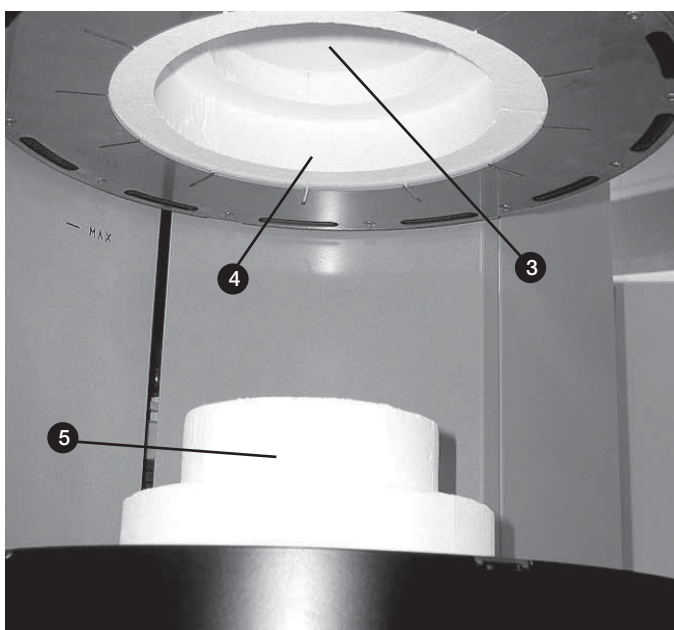

3.1 Base unit

The Cercon® heat plus P8 consists of an stable **metal housing (1)** and a **sintering chamber (2)**. The heat inside the sintering chamber is generated by molybdenum disilicide heating elements. At higher temperatures, these heating elements form a silicon oxide layer that protects them from corrosion and ensures their longevity. The heating elements are protected from contact with the firing objects by the **retort tube (3)** made of plasma ceramics. The walls of the sintering chamber are filled with high-quality **insulation material (4)** to keep the heat loss to a minimum. This results in an energy-saving process characterized by low thermal conductivity and heat storage. The design allows **maximum temperatures** of approximately **1,600°C**, evenly distributed throughout the sintering chamber. The heat insulation inside the walls of the sintering chamber consists of ceramic fibers.

The sintering trays (see Scope of delivery) are placed in the middle of the **insulated floor (5)**, from where they are lifted into the sintering chamber by the **lifting table (6)**.

Important note: Note that the sintering trays included with the Cercon® heat plus are suitable for use at the temperatures prevailing within the unit. The use of other sintering trays (such as those intended for use with e.g. Cercon® heat etc.) is not permitted and may result in damage to the unit. The lifting table is moved up and down by way of a manually operated electrical spindle drive.

The lifting table contains an electric fan, with the air taken in and removed via the **ventilation slits (7)**. The top of the sintering chamber features an **exhaust tube (8)** that also serves to optimize the exchange of air.

The **control panel (9)** and **display (10)** are ergonomically located at the front of the unit. This allows you to activate the four predefined sintering programs and to check the device parameters (see Controls). All processes are controlled by an electronic microcontroller inside the unit.

The unit rests on **adjustable feet (11)** for maximum stability.


The **keyhole for emergency lowering (12)** is located at the top of the unit and is actuated with the included key. (see Lifting table, emergency lowering).

Important: Use emergency lowering only in case of electronic failure! Pull the power supply plug before use the emergency lowering!

3.2 Controls and connectors

The following controls and connectors are available:


- Primary power receptacle (1)
- Power switch ON/OFF (2)
- Rocker switch for lifting/lowering the lifting table (3)
- Control panel (4) with membrane keyboard and display (see Control panel with membrane keyboard and display)
- Interface power supply with RS 232 interface module via D-Sub 9-pol plug (5) (only for Service).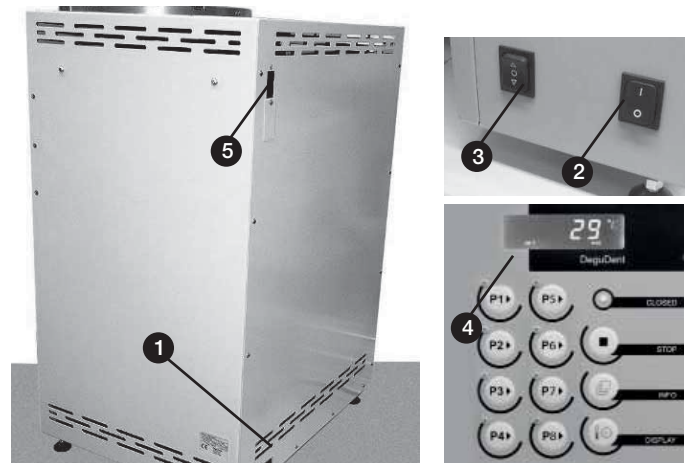

3.3 Scope of delivery

The scope of delivery includes:


- Cercon® heat plus P8 sintering oven
 - Power cable
 - Instructions for Use
 - 1 emergency lowering key for lifting table (7), see Lifting table, emergency lowering
 - 3 sintering trays (8)
- Important note:** Cercon heat sintering trays may not be used!
- 3 packs of sintering balls (9)
 - 1 exhaust tube (pre-mounted) (10)
 - 1 sintering block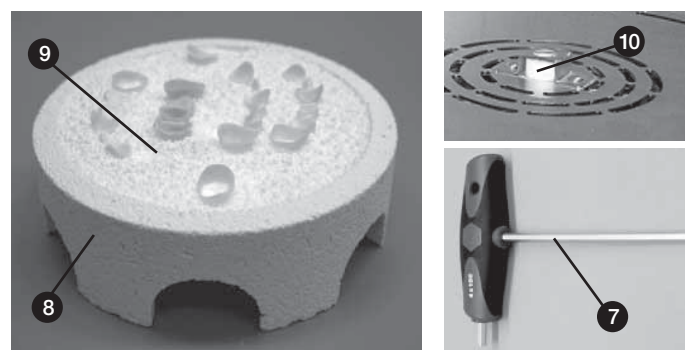

3.4 Technical specifications and operating environment

Power supply:	220 – 240 V, 50/60 Hz
Maximum power consumption:	3.0 kW
Protection class:	IP 20
Thermal protection class pursuant to EN 60519-2 without safety control:	Class 0
Dimensions (W × D × H):	540 x 610 x 745 mm
Chamber dimensions (ø × H):	120 x 130 mm
Sintering chamber volume:	2 liters
Weight:	Equipment: Approx. 75 kg Device including packaging and accessories: Approx. 95 kg
Deployment:	Indoors only, on a non-flammable surface
Ambient temperature:	+ 0 till + 40 °C – preferably 20
Humidity:	5 – 90%
Noise level:	< 85 dB(A)
Sintered material:	Cercon® base Zirkonoxid
Heating elements:	Made of molybdenum disilicide
Maximum temperature:	1.600 °C
Time to T _{max} :	Approximately 90 minutes
Heater elements:	Type S
Display:	Illuminated LCD
Input device:	Membrane keyboard

4 Setting up the furnace

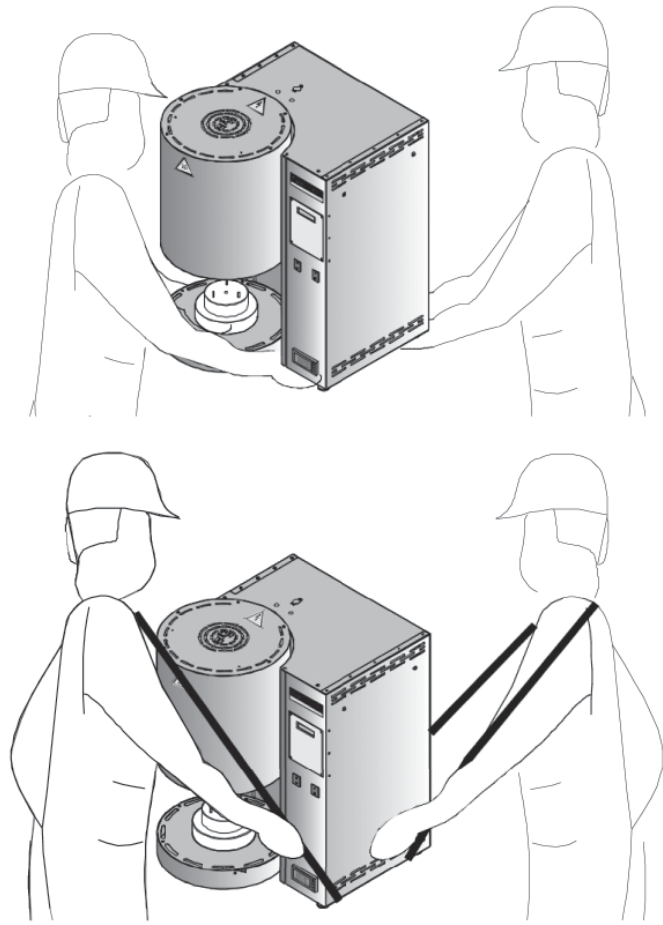4.1 Moving and unpacking

The carton containing the unit and the unit itself are very heavy (75 kg).

Use appropriate lifting aids.

Do not hold the unit by the lifting table when unpacking to avoid damage to the guide rails and spindle drive.

Observe the Moving and unpacking safety notes.

1. Move the package unit near the place where it will be deployed.
2. Open the cardboard case and remove the packaging material.
3. Remove the accessories and stored in a safe place.
4. With two people, seize the unit from underneath (do not lift by the lifting table) and carefully lifted out of the cardboard case (see illustration).
5. Use suitable carrying straps for further carrying if necessary (see illustration).
6. Check whether the scope of delivery is complete (see Scope of delivery) or damage in transit. Notify DeguDent GmbH immediately of any damage.

Note: Retain the packaging for later shipping or storing of the unit.

4.2 Setting up and connecting

Deploy the Cercon® heat plus P8 in a dry and well-ventilated room (IP20).

The table on which the unit is deployed must be capable of safely sustaining the unit's weight of 75 kg.

The surface of the table as well as the floor must be made of non-flammable materials.

Make sure to maintain a safety distance of 50 cm on all sides.

Before inserting the power cord, make sure that the power switch is in the OFF position.

Observe the Moving and unpacking safety notes.


Use only the sintering trays approved for use with Cercon® heat plus P8.

Note: Before turning on the Cercon® heat plus P8 for the first time, leave the unit to rest at the intended place of employment for at least **24 hours** to allow any moisture to evaporate from the electrical system.

In order to drive the insulating material and the sintering trays, the Cercon® heat plus P8 should first run program **P2** with sintering trays in place, but without sintering objects. During heating it can come to a smelling disturbance - this agitates from the binder discharge of the insulating material. The use of sintering trays other than those approved for use with Cercon® heat plus is not permitted.

We recommend to ventilate the location of the furnace during the first heating phase well.

1. Place the device in a dry room on a sturdy, non-flammable surface. The minimum distance from the nearest wall or object is 50 cm/19.69 in (see also Deployment and operation safety notes). If neighboring materials are certified as non-flammable, this distance can be reduced to 20 cm/7.87 in.
2. Use the adjustable feet to ensure that the Cercon® heat plus P8 is perfectly level.
3. Connect the device end of the power cord to the unit. Connect the wall plug of the power cord to a properly installed and protected wall outlet socket (see Technical specifications and operating environment). The Cercon® heat plus P8 must be the only device on its circuit.
4. Fill the sintering trays with sintering balls.

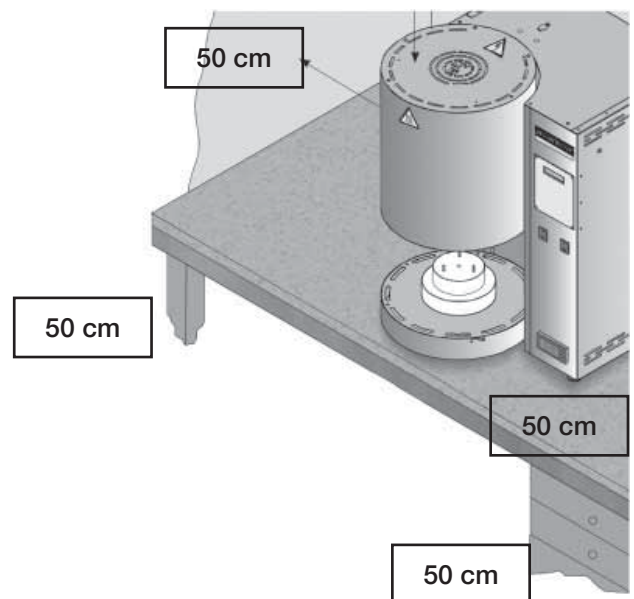5 Controls

5.1 Control panel with membrane keyboard and display

All programs can be started, functions selected, and parameters entered via the **control panel**. All relevant parameters are shown on the **display**. All entries are made via the **membrane keyboard** below the display. The control panel key and indicators serve the following functions:

P1–P8 keys

Start the heating programs.

Green P1–P8 key LEDs

Will be lit once one of these programs is started.

Red CLOSED LED

Will be lit once the lifting table has been lifted completely and the sintering chamber closed.

STOP key

Cancel a running program.

INFO key

Accesses the info menus
(see DISPLAY key, INFO key)

DISPLAY key

Accesses the temperature/time menus
(see DISPLAY key, INFO key)

Firing program

Program	Max. temperature	Duration	Material	Usage
1	1350 °C	ca. 8h	Cercon® base, colored, light, medium	1 Cercon® heat plus sintering tray
2	1450 °C	ca. 8h	Cercon® base, colored, light, medium	2 – 3 Cercon® heat plus sintering trays
3	1450 °C	ca. 12h	Cercon® base, colored, light, medium	large-span bridges > 8 units
4	1500 °C	ca. 8,5h	Cercon® ht	1 – 3 Cercon® heat plus sintering trays
5	1500 °C	ca. 14h	Cercon® ht	large-span bridges > 8 units
6	–	–	–	–
7	–	–	–	–
8	1580 °C	ca. 10h	Service firing	Serves the silicium oxide layer of the heating elements

5.2 General information on program execution

The Cercon heat plus P8 features **8 pre-programmed heating programs** (which can be modified only by authorized DeguDent service technicians) differentiated by their temperature/time profiles. The following programs are available:

P1 – Standard Cercon® base – Sintering at 1,350 °C, duration approximately 8 hours

P2 – Standard Cercon® base – Sintering at 1,450 °C, duration approximately 8 hours

P1 and **P2** are both suitable heating program for the Cercon® Zirconium application. During a full assembly of all three sintering trays (only Cercon® heat plus) the heating program **P2** is recommended

P3 – Sintering at 1,450 °C, duration approximately 13 hours (applicable to large-span bridges > 8 units).

P4 – Standard 1,500 °C, duration approximately 8 hours.

THIS PROGRAM IS RESERVED FOR FUTURE CERCON® MATERIALS IN 2011. DONT USE THIS PROGRAMM FOR CURRENT CERCON BASE PRODUCTS.

P5 – Sintering at 1,500 °C, duration approximately 13 hours (applicable to large-span bridges > 8 units).

THIS PROGRAM IS RESERVED FOR FUTURE CERCON® MATERIALS IN 2011. DONT USE THIS PROGRAMM FOR CURRENT CERCON BASE PRODUCTS.

P6 – Reserved

P7 – Reserved

P8 – Service firing, duration approximately 10 hours

The Service firing program **P8** serves the silicium oxide layer of the heating elements for the reconstruction. The Service fire program is used with the **empty furnace** in the following cases:

- After change of the heating elements by the DeguDent service technician.
- In case of framework-discolorations after very long use of the furnace.

Notice: The Service firing is accomplished with 1.580 °C and is not appropriate for the sintering of Cercon® Zirconoxid.

To start a program, the following **conditions** must be met:

- The temperature inside the sintering chamber is less than 200 °C.

- The display indicates that the lifting table was ready to be lifted by ("rel1") and has been lifted all the way. For your security the lifting table is driven manually into the final position.
- No program LED (green) is lit.
- No error message is shown (see Error messages and **troubleshooting**).
- The lifting table is lifted up

If the heating program has been started by pressing one of the **P1–P8**, program keys, the temperature control automatically implements the appropriate temperature profile. Starting the program also activates a safety contact for the oven heater that remains active until the program has terminated, unless a failure occurs during the process. In the event of a failure, the safety contactor for the oven heater turns off immediately (see Error messages and troubleshooting).

While the program is running, the calculated remaining **execution time**, including cooling time, is shown on the **display**. The **DISPLAY** key can be used to alternate between the execution time display and the **temperature display**.

While the program is running and until the chamber temperature has fallen below 200 °C, the **lifting table** motor is **locked** by relay #1 (displayed as "rel1") and cannot be lowered by pressing the **rocker switch**.

The unit has an integrated **temperature control function**. If the temperature does not increase within approximately 15 minutes, error message F10 is generated. If the temperature surpasses the maximum temperature of 1,600 °C, error message F70 is generated. During the holding time, the temperature is also controlled; in the event of a discrepancy of more than ± 10 °C, error message F80 is generated (see Error messages and **troubleshooting**).

Once the program has been started, the fan inside the lifting table is activated automatically. The fan turns off again once the program has terminated and the temperature has fallen below 80 °C. **Cooling** is effected as a natural process after the oven heater has been turned off. The cooling process continues until the temperature inside the sintering chamber has fallen below 200 °C. During the cooling phase, the current temperature is shown on the display. The unit of temperature can be set to °C or °F (see DISPLAY key, INFO key).


The program **terminates** as soon as the chamber temperature has fallen to 200 °C (with a hysteresis of 3 °C).

The green program key LED turns off, and relay #1 unlocks the lifting table, which can then be lowered manually. The end of the program run is indicated by an acoustic signal.

5 Controls

(5 seconds)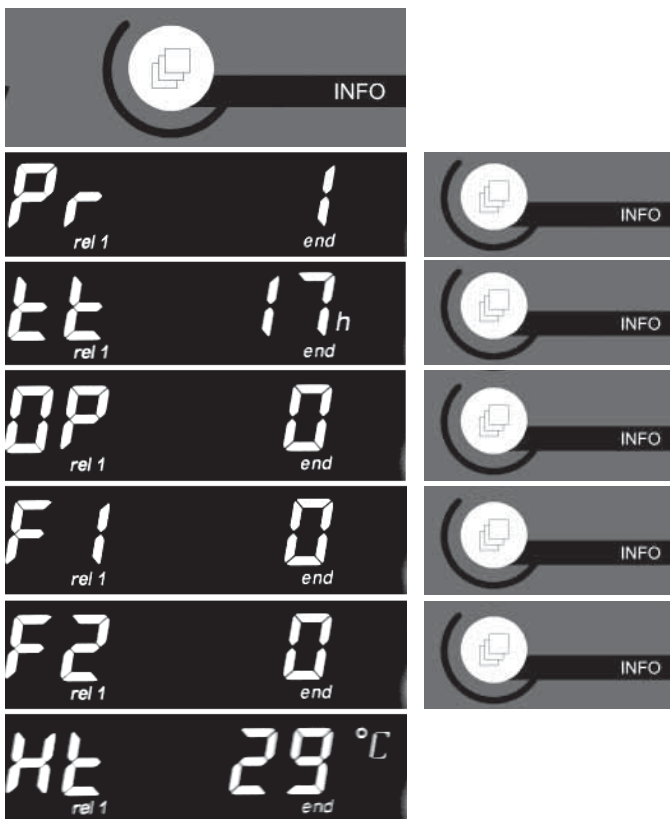

5.3 DISPLAY key, INFO key

The DISPLAY and INFO keys can be used to define additional settings or display information:

Alternating between °C and °F

Pressing the **INFO** and **STOP** keys together alternates the temperature display between °C to °F.

The temperature by the microcontroller/software processed exclusively in °C.

Alternating between execution time and temperature display

While the program is running, the **DISPLAY** key can be used to alternate between execution time and temperature display.

Additional information

Pressing the INFO key displays additional process-related information:

Current program number

Cumulative operating hours (max. 50,000 hours; 0 hours at delivery)

Current heating power in %

Error memory, last error displayed

Error memory, second last error displayed

Highest temperature since the start of the current program

If the **INFO** key is not pressed again for more than **5 seconds**, the original process information display resumes.

6.1 Turning on the unit and preparing the sintering trays


Always place the sintering trays (only Cercon® heat plus Sintertrays) in the middle of the insulated floor of the sintering chamber. Observe the **MAX** mark when stacking trays to avoid damage to the installation, the retort unit or the thermal element. The use of other sintering trays such as those intended for use with e.g. Cercon® heat or others is not permitted. Violation may cause damage to the insulation, the retort unit, the heating elements, or the thermal element.


Observe the Moving and unpacking safety notes.

1. Turn on the unit with the ON/OFF key. An acoustic signal is heard.
2. The display shows the message CH II, followed by the software version number and the current chamber temperature.
3. Load the sintering trays (only Cercon® heat plus Trays) with the Cercon® Zirkon-oxid frameworks and place them in the middle of the insulated floor of the sintering chamber.

Do not stack more than **three sintering trays** at a time, taking note of the **MAX** mark (which defines the maximum stacking height).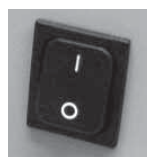

Acoustic signal (1 sec.)

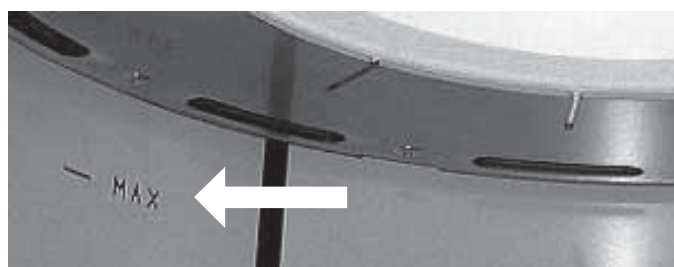6.2 Program selection and program start

1. Raise the lifting table bearing the sintering trays (only Cercon® heat plus Trays) as far as it will go in order to seal the sintering chamber. The **red Closed LED** will be lit.
2. Start one of the 8 predefined programs by pressing one of the **P1–P8** keys. To start program **P8** (Service firing), you must hold the **P8** key pressed for more than **5 seconds**.
3. The program number is briefly displayed, and an acoustic signal is sounded. The green program key LED is lit, and the fan inside the lifting table turns on.

till

(P8 Service firing/press fo 5 sec.)

Acoustic signal (1 sec.)

6 Operation

To start a program, the following conditions must be met:

- The temperature inside the sintering chamber is less than 200 °C.
- The display indicates that the lifting table was ready to be lifted by (“rel1”) and has been lifted all the way.
- No program LED (green) is lit.
- No error message is shown (see Error messages and troubleshooting).
- Lifting table is lifted up

Note: If the sintering chamber temperature exceeds 200 °C, the lifting table can no longer be lowered.

6.3 Canceling program execution

Program execution can only be canceled while the temperature is below 200°C.

To do so, press the **STOP** key. The heating process is discontinued, and the green program key LED turns off.

6.4 Program execution

While the program is running, the calculated remaining execution time is shown on the display (hh:mm format).

The display shows “rel2” to indicate that the heater has been turned on; during the heating process, the display also shows “heat.”

While the program is running, the **DISPLAY** key can be used to alternate between execution time and temperature display (see DISPLAY key, INFO key).

6.5 Program termination: Removing sintering trays

Burn hazard: Sintering trays (only Cercon® heat plus Trays) may be hot. Wear protective gloves. Observe the Moving and unpacking safety notes.

Once the holding time has run out, the heater turns off automatically, and the cooling phase begins.

1. During the cooling process, the current temperature is shown on the display.
2. The program is terminated as soon as the temperature has fallen to 200°C. The green program key LED turns off, and “rel1” indicates that the lifting table has been unlocked. The end of the program run is additionally indicated by an acoustic signal.
3. The fan is turned off automatically once the temperature has fallen to approximately 80°C.
4. Lower the lifting table all the way and remove the sintering trays (only Cercon® heat plus Trays).
5. To start a new program, proceed with step #1 under Program selection and program start.
6. If you are done the sintering, turn off the Cercon® heat plus P8 by turning the power switch to OFF.

Acoustic signal at 200 °C

Cleaning 7

The unit **does not require** any **maintenance**, but may have to be cleaned depending on the circumstances. Only the **outer walls** of the housing and the **external surfaces of the lifting table** need to be cleaned.

Use **regular commercially available water-based, non-flammable detergents** containing **no solvents**. When selecting a detergent, make sure that it does not damage the environmentally safe varnish with which the housing has been painted. Follow the instructions and safety notes of the detergent manufacturer.

Before cleaning the unit, allow it to cool to room temperature and disconnect the power cord. The insulated floor is adhesively attached and cannot be removed.

8 Error messages and troubleshooting

All repairs must be performed by authorized DeguDent personnel only.
 Only qualified electricians are allowed to open the unit and to work on electrical components.
 Disconnect the power cord before opening the unit.
 Any problems you cannot localize must be resolved by authorized DeguDent personnel only.
 When shipping the unit, please follow the Moving and unpacking safety notes.

8.1 Problems

Problem	Possible cause	Potential remedy
Control panel does not display anything	No power Control panels defective	Check connections and power cord as well as the appropriate fuses Notify service technician
Sintering chamber is not heated after the program is started	Error in program selection Unit has no power Defective heating element	Restart the program Check connections and power cord as well as the appropriate fuses Notify service technician
Sintering chamber heats very slowly	Defective heating element or thermal element	Notify service technician
Selected and temperature is not attained	Insufficient heating power due to insufficient voltage Defective heating element or thermal element	Notify service technician Notify service technician
Lifting table cannot be lowered	Temperature inside sintering chamber is above 200 °C Unit has no power Defective motor	Allow to cool for below 200 °C Check connections and power cord as well as the appropriate fuses Perform emergency lowering (see Lifting table, emergency lowering) and notify service technician)

8.2 Error messages

Various failure states and errors are shown in the display in the form of **error messages**, which starts with the letter **F** followed by **two digits**. The following table lists and explains these messages:

8.3 Cracks in the insulation or retort tube

Thermal expansion may cause cracks within the insulation over time. These do not adversely affect the function of the unit or the sintering results.

Smaller cracks inside the retort unit do not adversely affect the sintering result, either. In the event of larger cracks or chip parts, the retort tube has to be replaced (only by authorized DeguDent service technicians).

8.2 Error messages

Advertisement	Problem	Possible cause	Potential remedy
F10	Oven temperature does not rise	Defective oven heater	Notify service technician
F30	Measurement out of range	Problem with thermal element	Notify service technician
F31	Linearization	Problem with thermal element	Notify service technician
F40	Incorrect polarity of heating element		Notify service technician
F61	A/D converter timeout	System failure	Notify service technician
F62	Reference temperature below -20°C	Ambient temperature too low	Allow the unit to take on room temperature for 24 hours.
F63	Reference temperature above 70°C	Ambient temperature too high	Allow the unit to take on room temperature for 24 hours.
F64	Data memory error	System failure	Notify service technician
F65	Program memory error	System failure	Notify service technician
F66	EPROM defective	System failure	Notify service technician
F67	RAM defective	System failure	Notify service technician
F68	ROM defective	System failure	Notify service technician
F70	Tmax exceeded	$30^{\circ}\text{C} > 700^{\circ}\text{C}$ above segment temperature for 256 seconds	Notify service technician
F80	Holding time discrepancy	$\pm 10^{\circ}\text{C}$, holding time only	Notify service technician
F85	External failure	Input #2 module I/O closed	Notify service technician
F90	Power failure	Loss of power during program execution	

8 Error messages and troubleshooting

8.4 Lifting table, emergency lowering

If the lifting table cannot be lowered by the rocker switch during or after program execution, an **emergency lowering** of the lifting table may be performed manually. **Pull the power supply plug before use the emergency lowering!**

To do so, first pull the power supply plug before loosen the top cover of the Cercon® heat plus P8, insert the emergency key (included) into the keyhole and turn it to manually lower the lifting table (counterclockwise).

Notice: Use emergency lowering only in case of electronic failure! Pull the power supply plug before use the emergency lowering!

8.5 Power failure

The Cercon® heat plus P8 offers a **power failure protection feature**. Depending to the temperature the following steps have to be done:

- A power failure during the heating phase at a temperature of below 100 °C results in the program being aborted.
- A power failure during the heating phase at a temperature of above 100 °C results in the program being continued from the current temperature.
- A power failure during the holding time for the final temperature results in the program being aborted if the temperature was above 1000 °C at the time of the failure and the temperature had dropped by more than 50 °C at the time the power returned.

9 Disposal

This device is an electrical device pursuant to the German law on the marketing, return and environmentally compatible disposal of electrical and electronic equipment (ElektroG). The device has been labeled in compliance with the law and marked with the following symbol:

8.6 Returning the unit

When returning units to DeguDent **for repair**, please follow the instructions below:

- Units must be returned in their **original packaging**. Secure the Cercon® heat plus P8 well, Pat all protruding parts, and feel any hollow spaces inside the package with packaging material.
- Move the **lifting table** to the lower position before shipping. **Do not lift** the unit by the lifting table.
- If the unit is **damaged** due to **inappropriate packaging**, the **extra cost** incurred will have to be borne by the **sender**.
- When placing the unit in storage for some time, **protect it from temperature** extremes and from moisture. Always place the unit on a level surface to prevent distortion of the housing.
- The Cercon® heat plus P8 is generally shipped **without accessories** unless a technician expressly requests them.
- When shipping the unit, please include a **detailed description** of the problem and your contact data in case we have any questions.
- For units that are **out of warranty**, all shipping costs (both ways), packaging material, etc. will have to be paid for by the sender.

The device is not intended for private or home use. It has been produced and furnished for commercial use and must be disposed of properly by the last user pursuant to the regulation in the ElektroG.

Note: When disposing of the unit, observe the Moving and unpacking safety notes.

DeguDent GmbH
Rodenbacher Chaussee 4
63457 Hanau
Germany

We hereby declare the conformity of this product:

Cercon® heat plus P8

with the following directive(s):

RoHs – limitation of the use of certain dangerous substances in electrical and electronic devices, 2002/95/EC

WEEE – old electrical and electronic equipment, 2002/96/EC

Machine directive, 2006/42/EC

Low voltage directive, 2006/95/EC

EMC directive, 2004/108/EC

The following harmonised standards were applied:

DIN EN 746-1	(02/2010)
DIN EN 60204-1	(06/2007)
DIN EN 61010-1	(08/2002)
DIN EN 60519-1	(05/2004)
DIN EN 60519-2	(05/2007)
DIN EN 61000-6-2	(03/2006)
DIN EN 61000-6-4	(09/2007)
DIN EN ISO 12100-1	(04/2004)
DIN EN ISO 12100-2	(04/2004)
DIN EN ISO 61326	(05/2006)

Hanau, 28.03.2011
Dr. Udo Schusser
Head of Research and Development

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