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Cergo[®] vac

Vakuumpumpe

Instructions for use
Mode d'emploi
Istruzioni per l'uso
Instrucciones para el uso

DeguDent

A Dentsply International Company

Cergo[®] vac

D	Gebrauchsanweisung	5
GB	Instructions for use	7
F	Mode d'emploi	9
I	Istruzioni per l'uso	11
E	Instrucciones para el uso	13

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D

Vakuumpumpe Cergo[®] vac

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Cergo[®] vac vacuum pump

Application

Vacuum pump for the **Cergo[®]** porcelain furnace.

Technical data


Power supply:	refer to rating plate
Maximum power input:	130 W
Final vacuum:	Approx. 20 mbar
Ambient temperatures:	5 °C to 40 °C
Dimensions (mm):	300 (L) x 167 (W) x 197 (H)
Weight (kg):	8.5

For your safety

- **Read these operating instructions carefully before installing and operating this pump.**
Always comply with these operating instructions.
- **Only use the Cergo[®] vac vacuum pump with original spare parts and accessories.**
This is essential if the pump is to attain its quoted performance and be operated safely.
- **Do not open the housing. There are dangerously high voltages inside the housing.**
Repairs must only be carried out by authorised service technicians.
- **To prevent short-circuiting and damage to the unit, do not expose the Cergo[®] vac vacuum pump to dampness.**
- **The Cergo[®] vac vacuum pump must not be operated in potentially explosive areas.**

Installation

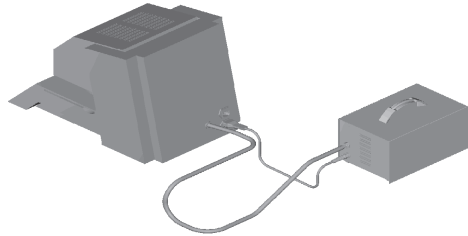- Place the Cergo® vac vacuum pump on a level, firm surface.
- Connect the vacuum hose and power cable to the Cergo® porcelain furnace and Cergo® vac vacuum pump. The vacuum hose is supplied with the porcelain furnace.

Ensure that the pump is ventilated properly while running. The ventilation slots in the housing must not be covered.

Your Cergo® vac vacuum pump is now ready for use.

It is controlled by the Cergo® porcelain furnace.

Care and maintenance

Check the pump, cables and hoses for signs of damage at regular intervals. This prevents accidents and breakdowns.

This vacuum pump does not require servicing or vacuum pump oil.

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DeguDent GmbH