DENTSPLY

DENTSPLY®

Conditioning Oven

OPERATION MANUAL

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MANUALE D'USO

Konditionierofen

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MANUAL DE INSTRUCCIONES

CE

SAFETY INFORMATION

Alert Signals

Warning

Warnings alert you to a possibility of personal injury.

Caution

Cautions alert you to a possibility of damage to the equipment.

Note

Notes alert you to pertinent facts and conditions.

Hot Surface

Hot surfaces alert you to a possibility of personal injury if you come in contact with a surface during use or for a period of time after use.

Warning

Warnings alert you to a possibility of personal injury.

This manual contains important operating and safety information. The user must carefully read and understand the contents of this manual prior to the use of this equipment.

Your DENTSPLY Oven has been designed with function, reliability, and safety in mind. It is the user's responsibility to install it in conformance with local electrical codes. For safe operation, please pay attention to the alert signals throughout the manual.

Warnings

To avoid electrical shock this oven must:

1. Use a properly grounded electrical outlet of correct voltage and current handling capacity.
2. Disconnect from the power supply prior to maintenance and servicing.

To avoid personal injury:

1. Do not use in the presence of flammable or combustible materials; fire or explosion may result. This device contains components which may ignite such materials.
2. Refer servicing to qualified personnel.

INTRODUCTION

Thank you for selecting DENTSPLY for your equipment needs. For maximum value and ease of start-up, please proceed as follows:

- Inspect the carton and contents for shipping damage. Notify the carrier immediately if damage is found.
- Verify that the complete unit has been received. Do not discard packing materials until all is accounted for.
- Read this Operation Manual thoroughly before deciding upon an appropriate location for the unit: you will want to consider the availability of power and other unit requirements, as well as user convenience.
- Insist that every operator of this unit becomes familiar with the Operation Section of this manual.
- Be sure to complete the Warranty Registration Card and return it to DENTSPLY.

If after reading this manual you should have any difficulties with the installation or operation instructions, please call: 800-786-0085 (or 1-717-845-7511).

DESCRIPTION / SPECIFICATIONS

DENTSPLY Conditioning Oven is a low-heat thermally controlled device for treatment of dental casts, materials, and appliances.

POWER REQUIREMENTS:	904979: 120VAC, 50/60 Hz, 420 Watts, 3.5 Amps 904968: 240VAC, 50/60 Hz, 408 Watts, 1.7Amps
TEMPERATURE RANGE:	From 5° C above ambient to 70°C (9°F above ambient to 158°F)
CHAMBER DIMENSIONS:	12" W x 12" D x 11" H (30 x 30 x 28 cm)
EXTERIOR DIMENSIONS:	13 1/2" W x 15 1/2" D x 19" H (34 x 39 x 48 cm)
SHELVES:	2 perforated aluminum shelves 11-5/8" x 11-5/8" (29.5 x 29.5 cm)
CHAMBER VOLUME:	1.00 cubic feet (0.028 m ³)
SHIPPING WEIGHT:	27 lbs. (12 kg)
UNIT'S ENVIRONMENT OPERATING CONDITIONS:	
POLLUTION DEGREE:	2
INSTALLATION CATEGORY:	II
ALTITUDE:	2000 Meters MSL (Mean Sea Level)
HUMIDITY:	80% maximum, non-condensing
ELECTRICAL SUPPLY:	120VAC or 240VAC
VOLTAGE TOLERANCE:	±10% of normal rated line
TEMPERATURE:	15°C to 40°C (59°F to 104°F)
PRODUCT USAGE:	This product is intended for use indoors only

INSTALLATION

LOCATION:

Place the unit on a level table or bench where it will be operated, away from drafts and wide variations in ambient temperature, near a power supply source that matches the unit service ID tag requirements. Allow 4" clearance around the unit for free air convection.

SHELF INSTALLATION:

Slide each of the two perforated aluminum shelves, supplied with the DENTSPLY Conditioning Oven, through the corresponding grooves provided in the side pilasters.

ELECTRICAL REQUIREMENTS:

120 VAC models require a 120 VAC, 50/60 Hz power source. They are supplied with a 3-wire line cord. It should be plugged into an outlet designed for 3-prong plugs. If an extension cord is used, it also should be the 3-wire grounded type.

240 VAC models require a 240 VAC, 50/60 Hz power source.

NOTE: LEAVE UNIT DISCONNECTED WHEN NOT IN USE.

MAINTENANCE

NOTE: MAKE NO ATTEMPT TO SERVICE OR REPAIR THIS PRODUCT UNDER WARRANTY BEFORE CONSULTING YOUR DEALER OR DENTSPLY TRUBYTE. AFTER THE WARRANTY PERIOD, SUCH CONSULTATION IS STILL ADVISED, ESPECIALLY WHEN THE REPAIR MAY BE TECHNICALLY SOPHISTICATED OR DIFFICULT.

IF ASSISTANCE IS NEEDED BEYOND WHAT THE DEALER CAN PROVIDE, PLEASE CALL DENTSPLY TRUBYTE CUSTOMER SERVICE AT (800) 786-0085 or 1-717-845-7511. NO MERCHANDISE, HOWEVER, SHOULD BE RETURNED DIRECTLY TO DENTSPLY WITHOUT PRIOR APPROVAL FROM DENTSPLY.

WARNING: DISCONNECT PLUG FROM ELECTRICAL OUTLET BEFORE ATTEMPTING ANY MAINTENANCE OR REPAIR OF THIS UNIT. REFER SERVICING TO QUALIFIED PERSONNEL.

ROUTINE CLEANING:

Clean up any spills immediately. Clean thoroughly between uses. Always unplug unit prior to cleaning. The chamber interior and shelves can be cleaned with a mild detergent or soap and water. Do not use abrasives or metallic scouring pads on any part of the unit.

To clean the viewing door use a soft damp cloth. A regular window cleaning solution can be used.

OPERATION

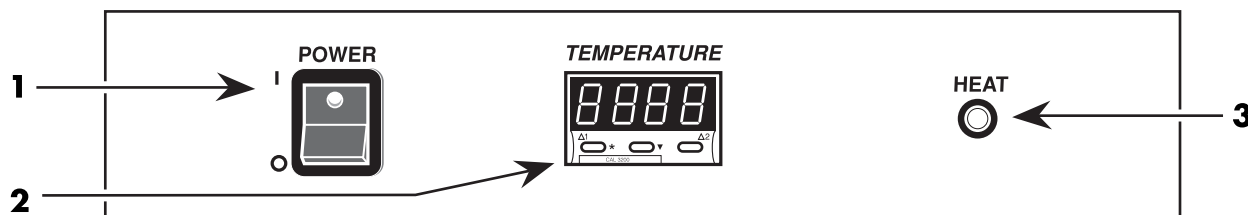
WARNING: DO NOT USE IN THE PRESENCE OF FLAMMABLE OR COMBUSTIBLE MATERIALS; FIRE OR EXPLOSION MAY RESULT. THIS DEVICE CONTAINS COMPONENTS WHICH MAY IGNITE SUCH MATERIALS.

START-UP:

1. Plug the line cord into an outlet that matched the specifications stated on the serial ID tag plate. Place the power switch in the **ON** position.
2. Set the desired operating temperature on the controller using the UP and DOWN arrow keys and allow sufficient time for the chamber temperature to stabilize.
3. Load the conditioning oven so that all chamber contents are spaced far enough apart to allow proper air circulation.
4. **IMPORTANT:** Door must be latched to maintain proper temperature.


Control Panel:

1. **POWER SWITCH:** Illuminates when conditioning oven is energized.
2. **TEMPERATURE CONTROLLER:** Microprocessor maintains chamber temperature.
3. **CONTROL STATUS LIGHT:** Amber light is on when controller is maintaining temperature.

OPERATION (con't)

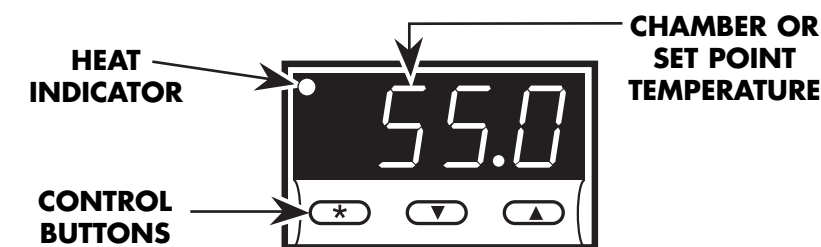TEMPERATURE CONTROLLER:

1. **CONTROLLER SELF-TEST:** When the conditioning oven is powered up, the controller will display **8888** along with the three decimal points and the heat **ON** indicator lamp. The display will then blank out for 2 seconds before showing the chamber temperature.
2. **HEAT ON INDICATOR:** The heat **ON** indicator lamp is lit when the chamber heaters are receiving power. The lamp will normally flash when the chamber temperature is at set point.
3. **SET POINT ADJUSTMENTS:** The temperature controller normally displays the chamber temperature. To view or change the temperature set point proceed as follows:

PRESS	CONTROLLER
★	View set point
★ ▼	Decrease set point
★ ▲	Increase set point

A. Press and hold the **"STAR"** (★) key and use either the UP or DOWN arrow key to adjust the set point to the desired temperature. Release the **"STAR"** (★) key.

B. Allow at least 30 minutes for the chamber temperature to stabilize.

OPERATION (con't)

TEMPERATURE CONTROLLER: (Con't)

TEMPERATURE CALIBRATION: (DO NOT OBSTRUCT VENT; CALIBRATE AT 55°C.)

- A. Place a calibrated digital thermometer in the approximate geometric center of the chamber.
- B. Press and hold the **"STAR"** (★) key and using the UP and DOWN arrow key, adjust the set point to the desired temperature.
- C. Allow the unit to run for at least 30 minutes.
- D. Controller display should now be indicating the set point temperature. Make note of the thermometer reading. If the controller display is different, follow the remaining steps.
- E. Press and hold both arrow keys until the controller display indicates **tunE**. Release the arrow keys. Press and release the down arrow key. The display should now indicate **LEUL**. Press and hold the **"STAR"** (★) key and using the **UP** arrow key, adjust the display to read **3**. Release the **"STAR"** (★) key. Press and release the **UP** arrow key until the display indicates **Zero**. The display should now alternate between **Zero** and a numerical value.
- F. Using the examples shown below and the thermometer value obtained in step above, enter the correct **Zero** value into the controller by pressing the **"STAR"** (★) key and using the **UP** or **DOWN** arrow key. If there is already a **Zero** value present then add the new value to the one already present.

Thermometer	=	50°C	Thermometer	=	60°C
Controller Reading	=	55°C	Controller Reading	=	55°C
<hr/>			<hr/>		
Subtract	=	-5°C	Subtract	=	+5°C

Enter **Zero** value of -5°C Enter **Zero** value of +5°C

- G. After the correct **Zero** value has been entered, press and hold the two arrow keys together until the display again indicates the chamber temperature. If the procedure was done correctly, the controller display should now agree with the thermometer reading to within ±0.5°C.
- H. Allow the unit to run for at least 30 minutes.
- I. Re-check the thermometer reading, the controller display and the thermometer should agree to within ±0.5°C. If not, repeat steps D, E and F above.

VENTILATION:

IMPORTANT: Vent must be unobstructed for optimum performance.

WARRANTY

The DENTSPLY Conditioning Oven is designed for dental use only. This warranty is not applicable to other uses. The DENTSPLY Conditioning Oven is warranted against defective material and workmanship for a period of 12 months from the date of purchase. Parts will be repaired or replaced at our option. This warranty extends only to the original purchaser and is subject to the following conditions:

1. The warranty registration card must be completed and returned to DENTSPLY within two weeks of date of installation.
2. Any servicing of the DENTSPLY Conditioning Oven must be performed by trained DENTSPLY Dealer Service personnel, or by DENTSPLY.
3. The DENTSPLY Conditioning Oven must not be subjected to abuse, improper installation, or application.
4. Damage due to improper return packaging will not be covered.

THERE ARE NO WARRANTIES, EXPRESS OR IMPLIED, WHICH EXTEND BEYOND THE DESCRIPTION ON THE FACE HEREOF. DENTSPLY neither assumes, nor authorizes any person to assume for it, any other liability in connection with the sale and use of the DENTSPLY Conditioning Oven.

DAMAGES ARE LIMITED STRICTLY TO REPLACEMENT OF THE DENTSPLY CONDITIONING OVEN. DENTSPLY EXPRESSLY DISCLAIMS LIABILITY FOR INCIDENTAL AND CONSEQUENTIAL DAMAGES RESULTING FROM THE USE OF THE DENTSPLY CONDITIONING OVEN.

Claims covered by this warranty will be honored when presented through your DENTSPLY Dealer within one month from the discovery of the defect. For more information, call 1-800-877-0020, ext. 4319 or 4581, or 1-717-845-7511.

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DENTSPLY International Inc.
DeguDent GmbH
Printed in U.S.A. Form No. 4283-D Rev. F 06/04