

Quick Start Instructions for your Multimat® Easy

GB

For more detailed information, please consult the Instructions for Use.

#	Action	Step 1	Step 2	Step 3	Step 4	Step 5	Step 6
1	Switch on/off	Toggle switch ON/OFF					
2	Stand-by	Push on/off					
3	Lift up/down	Push drücken					
4	Select program	Push drücken	Push to enter program number	Confirm with Enter			
5	Programming	Push Enter	With cursor buttons select firing parameters	With buttons or change parameters	Confirm with Enter		
6	Save program	Push	Push to enter Program number	Confirm with Enter			
7	Program start/stop	Push start/stop					
8	Copy program	Push	Push to enter program number of the program to be copied	Confirm with Enter	Push	Push Programmnummer eingeben	Confirm with Enter
9	Change parameters during firing or for one bake	Push Enter	With cursor buttons select firing parameters	With buttons change firing parameters	Confirm with Enter		
10	Menu settings	Push Menu	With cursor buttons select menu and push Enter	With cursor buttons select sub-menu and push enter	With cursor buttons or enter parameters	Push Enter	Push as often as program reappears
11	Activate quick cooling	Push					
12	Deactivate quick cooling	Push					
13	Confirm information	Push for No Push für Yes					