

October 2011

## **Installing the Multimat 'Easy Software 2.0.11 incl. SP2':**

Use a standard computer to download the Multimat 'Easy Software 2.0.11 incl. SP2' from the DeguDent website

Unpack the files "easy.2.0.11sp2\_20110922.tar" and "init.sh" from the archive file "easy 2.0.11 incl.SP2\_2011\_09\_22.zip" to an empty USB Drive.

Connect the USB Drive to the furnace.

Switch on the power.

As soon as the installation is done the software starts and shows "Multimat Easy V 2.0.11sp2(110922)".

Remove USB drive from furnace.