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# Multivac<sup>®</sup> eco

Instructions for use  
Mode d'emploi  
Istruzioni per l'uso  
Instrucciones para el uso

# Multivac<sup>®</sup> eco

<b>D</b>	Gebrauchsanweisung . . . . .	5
<b>GB</b>	Instructions for use . . . . .	12
<b>F</b>	Mode d'emploi . . . . .	19
<b>I</b>	Istruzioni per l'uso . . . . .	26
<b>E</b>	Instrucciones para el uso . . . . .	33

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- 1 Working with the Multivac eco** ..... 13
  - 1.1 Contents ..... 13
  - 1.2 Technical Data ..... 13
  - 1.3 For your own safety ..... 13
  - 1.4 General operating notices ..... 14
    - 1.4.1 Proper use ..... 14
    - 1.4.2 Control panel ..... 14
  
- 2 Getting started** ..... 15
  - 2.1 Setting up the unit ..... 15
  - 2.2 Operation ..... 15
  
- 3 Care and Maintenance** ..... 16
  - 3.1 General notices ..... 16
  - 3.2 Changing the filter ..... 17
  
- 4 Problem solving** ..... 17

# 1 Working with the Multivac eco

## 1.1 Contents of delivery

- Basic Multivac eco unit
- Mains supply cable
- Reserve filter
- Instructions for use
- Drilling template
- Mounting material

## 1.2 Technical Data

Required Voltage:	220–240 V / 50 Hz
Maximum power consumption:	150 W
Required fusing:	2 AT
Environmental temperature:	4 °C to 40 °C
Dimensions (mm):	520(H) x 385(W) x 320(D)
Weight:	approx. 8.8 kg

## 1.3 For your own safety


- !** Please read all instructions carefully before installing and operating this machine. All **■** warnings or notices contained within these instructions should be followed at all times.
- !** This Multivac eco unit should only be used together with original spare parts and **■** accessories. Only in this way can the working capacities indicated be reached whilst upholding the required safety standards.
- !** Do not open the casing. On the inside of this unit there are dangerously high voltages at **■** hand. Authorised service personnel should only carry out Service and repair work.
- !** Protect the unit from condensation build up, preventing short-circuiting and damage to **■** the machine.  
If the unit is not being used for longer periods, please disconnect the mains supply.

## 1.4 General operating notices

### 1.4.1 Proper use

- Mixing of plaster and investment materials under vacuum conditions
- Mixing of silicone based duplicating materials or reversibles

### 1.4.2 Control panel

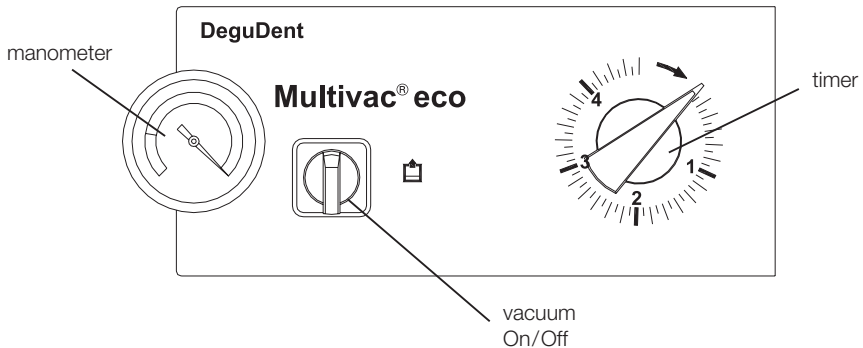#### ***Vacuum On/Off***

This switch allows you to turn the vacuum pump either on or off manually.

- ! **When evacuating the unit, please make sure none of the material being mixed is sucked into the vacuum system.**

#### ***Timer for the mixer***

Choose the length of mixing time by simply turning the timer to the value required. The mixer starts automatically.

- ! **Complete separation of the unit from the mains is only achieved when the mains supply cable is disconnected from the wall socket.**
- ! **In order to ensure homogenous mixing of the materials, we recommend using unscratched vessels.**

## 2 Getting started

### 2.1 Setting up the unit

- Using the supplied drilling template, drill three holes and insert the wall plugs.

**!** Please assure there are no water or  
**■** electricity lines in the wall before drilling.

- Screw the top two screws into the holes you have made.
- Hang the Multivac eco onto these screws.
- Secure the unit by screwing in the third screw, which is located at the lower column.
- Connect the unit to the mains supply.  
**The socket for the mains connection is located on the right hand side of the rear of the unit.**

### 2.2 Operation

- Fill the mixing flask with the materials you wish to stir.  
**Please do not fill the flask above the maximum filling level shown on the side.**  
**When mixing powder and liquid together, it is advisable to stir the two components well with a spatula beforehand.**
- Remove any remnants of powder from the flask rim and set the stirring section on top.
- Press the mixing flask up under the coupling and turn on the vacuum pump.  
**Hold on to the mixing flask until the vacuum level is high enough to hold the flask in position alone.**  
**Keep an eye on the material in the flask. If it starts to rise up inside the flask quickly, and reaches a point about 1 cm below the rim, then you should turn on the stirring mechanism to avoid material being sucked into the vacuum system.**
- Start the stirring mechanism by turning the timer to the required value.  
**After the set time has elapsed, stirring will stop automatically.**
- Hold on to the flask once more and turn off the vacuum pump.  
**Please remember that the flask is not held in position once the vacuum has been turned off.**

### 2.3 Mixing of Silicones

- For the mixing of silicones, it is imperative not to exceed the maximum filling of the mixing vessels (small vessel max. 100 g, large vessel max. 400 g). Mixing duration is 75 secs.

## 3 Care and Maintenance

### 3.1 General notices

Care and maintenance involves all procedures which keep the Multivac eco in best working order.

Check the machine for mechanical faults or damage regularly. In this way, you can prevent accidents and fall out time.

Remove all dirt or soiling from moving parts immediatly. This ensures they remain moveable and function properly.

Mixing motor and vacuum pump are maintenance free.

**Before cleaning,**

**the unit should be turned off and disconnected from the mains.**

**Make sure that no liquids enter the unit during cleaning. In this way you avoid the risk of**

**short circuiting which could lead to a fire or cause the unit to break down completely.**

**On the inside of this unit there are dangerously high voltages at hand. Service and repair**

**work should only be carried out by authorised service personnel.**



**Manipulation of the unit by unauthorised persons reduces the safety of the unit and**

**endangers surroundings and other persons operating it.**

### 3.2 Changing the filter

Filter changing becomes necessary when the amount of time taken when evacuating or flooding the flask becomes noticeably longer than usual.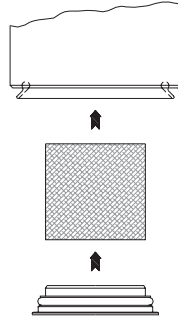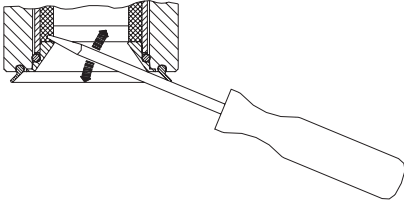

- Carefully remove the sealing ring from the coupling with a screwdriver.
- Pull out the filter element.
- Remove any remnants of material from the coupling.
- Set the filter on top of the sealing ring.
- Replace the filter along with the sealing ring back into the coupling. The sealing ring must audibly click into place.

**!** Operating this unit without a filter can seriously damage the machine and is not to be  
**■** undertaken at any circumstance.

## 4 Problem solving

*When turning on the unit or starting mixing, the fuses blow.*

The fuse section is located next to the mains socket. Repeated fuse cut out points to a more serious problem within the machine which should be dealt with by service technicians. Cut outs which occur once only could have been caused when the stirring motor is blocked. In this case, the blockage should be removed and the fuse replaced, after which the unit can be started up once more.

**!** Before attempting to change a blown fuse,  
**■** the unit must be disconnected from the mains.

# Declaration of Conformity

EU Declaration of Conformity, EU Machines Directive 98/37/EG

DeguDent GmbH

Postfach 13 64  
D-63403 Hanau

We hereby declare that the:

## Multivac® eco/stirring and evacuating unit

conforms with the basic requirements of the following EU directives:

1. **EU Machinery Directive 98/37/EG**
2. **EEC Directive on electrical equipment designed for use within certain voltage limits 73/23/EWG**
3. **EEC Directive relating to electromagnetic compatibility 89/336/EWG**

The following harmonized standards were employed:

**EN 292-1: 1991**

**EN 292-2: 1995**

**EN 60204-1: 1998**

**EN 61010-1: 2002**

**EN 50081-1: 1993**

**EN 50082-1: 1997**

Hanau, 10.10.2003

Hanau, 09.10.2003

Torsten Schwafert  
Director business area  
dental laboratory products

Dr. Udo Schusser  
Research & Development

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