

May, 25th, 2011

Instructions for Download Program Updates

Update (Update Duceragold love.zip) for ProFire ceramic furnaces:

- Firing programs Duceragold love
- Correction of non precious alloy firing programs Duceram love

Unpack the Zip folder and save the update file in the highest level (not in a folder) of the memory stick, supplied with the furnace. Use this stick only!

Attach the stick to the furnace and switch the furnace on. During the start procedure the update starts automatically, afterwards a self test takes place. As soon as the furnace shows the main menu, you can remove the stick and restart the furnace.

Control, if the new programs are installed in the preinstalled programs.

The program update has neither influence on the software nor on existing individual programs.