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Multicast[®] Printer

Instructions for use
Mode d'emploi
Istruzioni per l'uso
Instrucciones para el uso

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Multicast[®] Printer

Scope of delivery

The scope of delivery of the Multicast Printer includes:

1 printer module	1 spare paper roll
1 hexagon key	1 round cell 1.5V

Accessories available

spare ribbon REF 2351 1404	spare paper roll REF 2351 1405
----------------------------	--------------------------------

Technical data

Voltage supply	5 V (via Multicast compact)
Power consumption	10 W
Ambient temperatures	4 °C to 40 °C
Storage temperatures	-20 °C to 60 °C
Dimensions (h x w x d)	100 x 120 x 170 mm
Weight	approx. 22 N

General information

Carefully read these instructions for use prior to installation and putting into operation. The information contained in these operating instructions must be adhered to. The Multicast Printer must only be operated with original spare parts and accessories. Only in this way the performance data provided can be achieved and the required operating safety is ensured. Protect the unit against moisture to avoid short circuits and damage to the unit

Connecting the Multicast Printer

- Switch off the Multicast compact. Remove the protective cap from the 25-pole port (1) of the Multicast compact.
- Screw off the two fixation screws (2) and remove the cover plate (3).
- Fasten the Multicast Printer instead of the cover plate and connect the printer cable to the interface (port) (1) of the Multicast compact.

Inserting the battery

In order to print out date and time in the casting protocol, the clock of the Multicast compact must be activated.

The clock is activated by inserting the battery into the control element. This ensures that the clock will continue to run even after the unit has been switched off.

- For this purpose switch off the Multicast compact.
- Remove the two fixation screws (5) of the control element and remove the control element (6) by pulling it upward carefully.
- Insert the 1.5 V round cell into the battery case (7) of the control element. Please observe the polarity indicated in the battery case.
- Reinsert the control element into the control pillar (8) and screw it home with the two fixation screws (5). Make sure that the connecting cable (9) will not be wedged in while inserting the control element.
- Set date and time in accordance with the information provided in the operating instructions of your Multicast compact.

Putting the printer into operation

Connect the Multicast Printer to the Multicast compact as described in chapter 6. Switch on the Multicast compact.

The Multicast Printer will automatically switch to operation mode. The operation mode is indicated by a green LED. After each casting process operation mode of the printer can be activated or deactivated at any time. With the paper feed button „LF” the paper feed is switched on or off. For this purpose the printer must be offline.

Moving the control element

The Multicast Printer is prepared to receive the control element. Moving the control element from the control pillar to the Printer allows the use of the Multicast compact even under a low rack system. If you would like to move your control element, the following steps are required:

- Prior to moving the control element switch off your Multicast compact and unplug the unit.
- Loosen the two fixation screws (10) of the cover (11) and remove the cover by gently pulling it upward.
- Remove the two fixation screws (5) of the control element and pull out the control element (6) from the control pillar (8).
- Pull off the plug of the connecting cable (9) from the control element and place the cable into the control pillar (8).
- Close the control pillar (8) with the printer cover (11). Make sure that the connecting cable will not be wedged in when inserting the cover.
- Screw the two fixation screws (5) in the control pillar again.
- Connect the plug of the connecting cable (9) with the control element (6) and insert the control element into the Printer. Make sure that the connecting cable will not be wedged in when inserting the control element.

Exchanging the paper roll

- Connect the printer as described in chapter 6 and switch on the Multicast compact.
- Deactivate operation mode of the Printer. For this purpose press the „SEL”-button (12) so that the green LED goes out.

- Activate the automatic paper feed by pressing the paper feed button „LF“ (13).
- Remove the housing by turning the locking button and pull off the printer to the front.
- Thread the new paper roll in the slot (15).
- Cleanly cut beginning of the paper roll and correct direction of unwinding must be ensured when inserting the paper.
- If the paper is fed beyond the tear-off edge, deactivate the automatic paper feed by pressing the paper feed button „LF“ (13) again and place the paper roll into the paper holder (16).
- Slide the housing back onto the support a) and lower the housing b). Make sure that the housing correctly engages on the locking button (14).

Exchanging the ribbon

- Remove the cover (17) of the Multicast Printer.
For this purpose carefully press the bottom of the cover to the front with a screwdriver.
- Remove the old ribbon by pressing on the lower left corner of the ribbon cartridge (18).
- Insert the new ribbon cartridge. First thread in the left and then the right side. The paper must be between the ribbon and the cartridge.
- Put on the cover again.

Elimination of errors

After casting no casting protocol is printed out:

- **Check whether your Printer has been connected correctly.**
When switching on the Multicast compact the green LED of the Printer should be automatically switched on. The operation mode can be activated or deactivated by pressing the „SEL“ button.
- **Check whether the casting protocol has been activated in your Multicast compact.**
The presetting for the casting protocol must be set to „intern (internal)“. Chapter 7 of the operating instructions of your Multicast informs you how to activate the casting protocol.
- **Check if your printer functions properly.**
Activate the automatic test function of the Multicast Printer by pressing down the „SEL“ button for a few seconds. The printout starts as soon as the button is released.
- **The density of the printout is decreasing.**
- **Check the ribbon.**
Chapter 11 informs you how to exchange the ribbon.

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DeguDent GmbH