

# Application reference

Solaris® gold bath supra can be used in a GAMMAT easy unit.


No activator is required.

The amount of Solaris gold bath supra required for precipitation can be determined by checking the display of the GAMMAT easy unit.


All other device parameter settings can be determined from the corresponding original instruction manual of the unit.

Recycling:

For the most effective deposition of the remaining gold we recommend to run a triple cycle in the recycling mode.

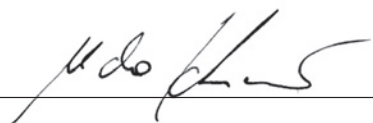

Dr. Udo Schusser  
Director, Research and Development

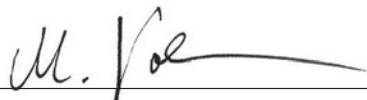

Dr. Markus Vollmann  
Director, Application Technology