

Application reference


Solaris® gold bath supra can be used in Genius-PERFEKT trend galvanano units. It is suitable for multiple precipitation cycles.

No activator is required.


Enter the following bath parameters for 500 ml of Solaris® gold bath supra:

| | |
|----------------------|--------|
| Bath's gold content: | 7.50 |
| Bath volume: | 0.500 |
| Bath type: | NORMAL |

All other device parameter settings can be determined from the corresponding original instruction manual of the unit.

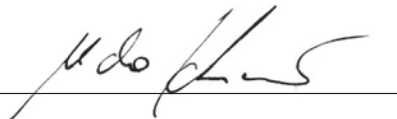

Dr. Udo Schusser
Director, Research and Development

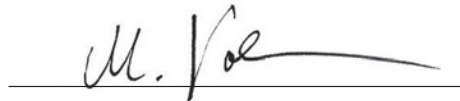

Dr. Markus Vollmann
Director, Application Technology