SOLARIS

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Instructions for Use

Solaris Conductive Silver Varnish Airbrush (and airbrush set cleaner)

Solaris Conductive Silver Varnish Airbrush is required for creating an electrically conducting contact surface on the objects to be electroplated using an airbrush gun. The cleaner is suitable only for cleaning the respective airbrush gun.

Procedure: Shake Solaris Conductive Silver Varnish Airbrush well before use (at least for two minutes). The integrated mixing ball should be easy to hear while shaking. Fill the filling chamber or screw the bottle with Solaris Conductive Silver Varnish Airbrush directly onto the Sirius XY. The primary coping or stone dye to be electroplated must be free of dust, clean, completely dry and **free of grease**. Apply **Solaris Conducting Silver Varnish Airbrush** evenly in a single thin layer. For a better connection to the cathode rod, manually paint on the varnish onto the contact area between the object and the cathode rod. The airbrush gun facilitates perfect application of Solaris Conductive Silver Varnish Airbrush and ensures excellent galvanic deposition, minimal joint gaps and high adhesive bond strengths.

Caution: Before electroplating, allow Solaris Conductive Silver Varnish Airbrush to dry at room temperature for at least 30 minutes. If this is not observed, the gold bath might be contaminated, and the electroplating result could be compromised.

Protective measures: The conducting silver varnish may soil and stain surfaces and clothes.

Special risks: Solaris Conducting Silver Varnish Airbrush is highly flammable. Solaris cleaner for Airbrush is flammable. Keep away from sources of ignition - no smoking. Do not inhale the spray. Avoid contact with the skin. In case of contact with the eyes, rinse immediately with plenty of water and seek medical advice. Take precautionary measures against static discharges. In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible). Use only in well-ventilated areas. If swallowed, do not induce vomiting; seek medical advice immediately and show the container or label.

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DeguDent GmbH