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**Instructions for Use Solaris® Aktivator**

**Solaris Aktivator** is a lustre agent additive for the Solaris gold solution for galvanofforming processes.

**For dental use only!**

**Attention:** Shake Solaris Aktivator well before each use and check for crystal formation. If crystalline particles are visible in the activator, do not continue to use it.

**Processing: Solaris Aktivator** is added to the gold solution at a ratio of 1:20 (e.g. add 5 ml of activator to 100 ml of gold solution). The quality of the precipitation depends on the precise dosage of liquid and activator and the correct power settings on the device. A measuring cylinder is recommended for dosage.

**Solaris Aktivator** should be used in connection with the Solaris gold solution only. Do not use other chemicals to avoid undesirable chemical reactions.

Amount of liquid: 50 ml, Color of liquid: White

**Protective measures:** No special protective measures are required when working with **Solaris Aktivator**.

**Storage:** Avoid contamination, direct sunlight and extended exposure to oxygen. Close container carefully after use. Store the liquid in a cool place, protected from sunlight, preferably in a cabinet designated for chemicals.

**Disposal: Solaris Aktivator** must be disposed in accordance to applicable laws and regulations.

**Special risks:** Harmful by inhalation and if swallowed. · Keep container tightly closed and in a well-ventilated place. · When using do not eat or drink. · In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. · Wear suitable protective clothing, gloves and eye/face protection. · In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

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