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Instructions for Use Solaris gold solution supra

GB

The **Solaris gold solution supra** is a non-cyanide gold sulphite solution, to be used without an activator. With **Solaris gold solution supra**, galvanic precipitation (electroplating) can be achieved for dental use (for inlays, crowns, bridges, and secondary constructions). The solution is particularly appropriate for the 2^o Weigl technique.

Contraindications: Do not use in case of confirmed hypersensitivity to gold, or copper.

Side effects: Allergies to metals contained in the liquid as well as to electrochemically caused dysaesthesia are possible. Individual cases have been reported of systemic side effects caused by metals contained in the liquid.

Interactions: Avoid occlusal and approximal contact between different metals.

It has been observed in clinical applications that discolorations may occur if the silver conductive lacquer has not been thoroughly removed from the galvanic caps.

Protective measures: Develops toxic gases when it comes into contact with acids. If gases get into eyes, rinse thoroughly with water and consult a doctor. Do not mix with acids. The gold solution may soil and stain surfaces and clothes. Take all the usual precautions when handling chemicals

Instruction for processing:

Attention: The **Solaris gold solution supra** can be used only once. **Solaris gold solution supra** is not compatible with dental gypsum. Please use modelling plastic instead. Avoid impurification and mixing the gold solution with other materials. They will affect the electroplating result negatively or even make the solution unusable. Any objects dropped in the bath must be removed with plastic tweezers only. To avoid contamination of the gold solution, close the bottles immediately after use and store in a cool place, well protected from sunlight.

Dosage: The amount of gold solution used will depend on the size of the objects to be electroplated. The power settings and layer thicknesses are set on the device using the Solaris model for parameter determination. The Solaris unit will automatically calculate the amount of liquid needed.

Amount of liquid	500 ml	Color of precipitation	Saturated yellow, silk matte to high lustre
Gold content (net usable).....	6,75 g	Time for precipitation/layer thickness.....	5 h/200 µm – 6 h/250 µm – 7 h/300 µm
Working temperature.....	Around 65 °C	Movement in bath.....	Mandatory
Color of liquid	Colorless and clear		

Both liquid quantities and activator dosage: The layer thickness (µm) and the surface of the objects to be electroplated will determine the amount of gold bath to be used. The power settings are determined by the Solaris model for parameter determination. Once the parameters have been set on the unit, the amount of liquid required for the process is displayed. Please ignore the activator quantity display on the Solaris unit.

Storage: Avoid contamination, direct sunlight and extended exposure to oxygen. After use, keep container tightly closed and in a well-ventilated place, well protected from sunlight.

Shelf life: See label.

Recycling: Effected by precipitation cathode or by the recycling sponge. See instructions for use.

Disposal: Any product residue must be disposed of in accordance with applicable laws and regulations.

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DeguDent GmbH