# Selectaplus H / Trevalon / Universal CLEAR

## MATERIAL

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# Selectaplus H / Trevalon / Universal CLEAR

## MATERIAL

Hot-curing polymers consisting of powder and liquid. The powder is based on polymethyl methacrylate and the main component of the liquid is methyl methacrylate.

Selectaplus H / Trevalon / Universal CLEAR are according to DIN EN ISO 1567:2000, Type I, Class 1.

Selectaplus H / Trevalon / Universal CLEAR are cadmium-free.

## APPLICATIONS

For fabricating full and partial dentures.

## PROPERTIES

Selectaplus H / Trevalon / Universal CLEAR are color-stable and exhibit outstanding physical properties.

## DIRECTIONS FOR USE

### Preparations:

Boil out the wax carefully. Use a clean paintbrush to apply separating agent (e.g. Isolant) to all plaster areas and allow to dry. Repeat. Surface must be thoroughly dry, allow no residue to remain in the crevices.

### Mixing:

**Selectaplus H / Trevalon:** The mixing ratio is 24 g of powder to 10 ml of Universal Denture Liquid.

**Universal CLEAR:** The mixing ratio is 22 g of powder to 10 ml Universal CLEAR Liquid.

Free dosage is possible.

1. Pour liquid into a mixing bowl.
2. Add powder until an excess appears on the surface.
3. Mix immediately.

### Processing Times:

Processing times are dependent on the room temperature and the quantity of material mixed. The following times assume a temperature of 23 °C.

|                  |                                     |
|------------------|-------------------------------------|
| Mixing time:     | 1 minute                            |
| Doughing time:   | 10 minutes (from completing mixing) |
| Processing time: | 60 minutes                          |

### Pressing:

Ensure that the temperature of the flask is approximately 23 °C. Pack the dough into the mold. Cover with a thin polyethylene sheet. Assemble flask and proceed with a trial closure. Open flask, add material if necessary and proceed with another trial closure. Remove separating sheet, cut away flash, and clamp flask into holder.

### Polymerization:

#### Method 1.

1. Place flask in cold water.
2. Slowly raise to boiling point (within 30 minutes).

3. Continue boiling for a further 30 minutes.

4. Allow to cool slowly in water.

#### Method 2.

For curing thick dentures only.

1. Leave the flask in the press for approx. 1 hour.
2. Place in warm water (70 °C) for 30 minutes.
3. Raise temperature to 100 °C within 15 minutes.
4. Boil for 30 minutes.
5. Bench cool.

“CAUTION, do not place the flask in contact with any source of heat”.

### Repairs:

To obtain a maximum repair strength it is recommended that repairs or additions be made using the denture base that was employed initially and using one of the recommended curing cycles. An autopolymerizing resin such as Selectaplus for repairing Selectaplus H or Rapid Repair for repairing Trevalon could also be used.

## STORAGE INSTRUCTIONS

Keep in a cool, dry place (max. 25 °C).

Keep containers closed when not in use.

Avoid direct sunlight.

## SAFETY WARNINGS

for Universal Denture Liquid and Universal Clear Liquid

|   |  |                                      |
|---|--|--------------------------------------|
| F   | Xi   | Liquid contains methyl methacrylate. |
|  |  |                                      |
| Highly Inflammable  | Irritant   |                                      |

R 37/38 Irritating to respiratory system and skin.  
R 43 May cause sensitization by skin contact.

S 24 Avoid contact with the skin.  
S 37 Wear suitable gloves.  
S 46 If swallowed, seek medical advice. immediately and show this container or label.

For dental use only.

### Contraindication:

Do not use this product for preventive reasons if patient has shown hypersensitivity to any of the components it contains.

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