Cercon® Malfarben

ht body-base und ht body-match

Leitfaden zur Farbgestaltung
Shade-matching guide
Mini-guide d'utilisation des maquillants
Guida alla riproduzione cromatica
Manual para la configuración del color

Cercon® Malfarben ht body-base und ht body-match

D	Leitfaden zur Farbgestaltung	4
GB	Shade-matching guide	7
F	Mini-guide d'utilisation des maquillants	10
I	Guida alla riproduzione cromatica	13
E	Manual para la configuración del color	16

Cercon® Malfarben ht body-base und ht body-match

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Cercon® ht body base/ Cercon® ht body match – Stains

Cercon® stains are suitable for applying individual shades and characterizations to zirconia (Y-TZP) crowns and bridges with a coefficient of thermal expansion of $10.5 \mu\text{m}/\text{m} \cdot \text{K}$ (25–500°C), preferably made of Cercon®.

Follow the Directions for Use for Cercon®ceram Kiss/Cercon®ceram love. For the most current versions of the Directions for Use and firing tables, visit www.degudent.de.

Cercon® ht body base/Cercon® ht body match shade-matching guide

The new Cercon® ht generation allows the fabrication of fully contoured frameworks. The appropriate shade is obtained using the new Cercon® ht body base/Cercon® ht body match stains in addition to the familiar Cercon® body stains modifying the pre-coloured Cercon® ht light/medium frameworks.

Staining with Cercon® ht light/Cercon® ht medium:

Cercon® body base A/B: As foundation for Cercon® ht (light/medium),
A or B shades.

Cercon® body base C/D: As foundation for Cercon® ht (light/medium),
C or D shades.

Cercon® body match 1: Stain mix for customizing A shades

Cercon® body match 2: Stain mix for customizing D shades

The framework should be clean and free of grease before staining.

Fabrication-related surface defects (e.g. cutter flutes) should be levelled to facilitate application of the stain and to avoid pooling and the formation of grooves or gaps.

Method:

1. Select Cercon® ht light or medium to match the desired tooth shade.
2. Air-abrade the completed framework using 110- μ m aluminium oxide at 3 bar of pressure.
3. Apply body base stain to match the desired tooth shade. Mix body base well with staining liquid (to a creamy consistency) and apply in a thin and even layer. Fire as for a shade-specific fixation/glaze firing.
4. Proceed with staining as per the shade-matching chart.

Note: body base and stains (e.g. amber, taiga, pink) should be mixed with a spatula tip of glaze to ensure an even lustre of the restoration; no additional glaze firing will be required.

Shade-matching chart:

Shade	Cercon® ht framework	Firing 1	Firing 2	Firing 3	Firing 4	Firing 5
A1	light	body base A/B	body match 1	-	-	-
A2	light	body base A/B	body match 1	-	-	-
A3	medium	body base A/B	body match 1	body match 1	-	-
A3.5	medium	body base A/B	body match 1	body match 1	-	-
A4	medium	body base A/B	body match 1	body match 1	-	-
B2	light	body base A/B	body 3	-	-	-
B3	medium	body base A/B	body 8	-	-	-
B4	medium	body base A/B	body 7	body 7	-	-
C2	medium	body base C/D	taiga	taiga	-	-
D2	light	body base C/D	body match 2	body base C/D	-	-
D3	medium	body base C/D	body base C/D	body match 2	-	-
D4	medium	body base C/D	body base C/D	body match 2	body match 2	-

The following shades can be obtained by mixing:

Shade	Cercon® ht framework	Firing 1	Firing 2	Firing 3	Firing 4	Firing 5
B1	light	body base A/B	3x body base A/B* 1x pink* 1x taiga*	-	-	-
C1	light	body base C/D	2x body base C/D* 1x grey* 4x taiga*	-	-	-
C3	medium	body base C/D	2x body base C/D* 1x grey* 4x taiga*	2x taiga* 2x body 7*	2x taiga* 2x body 7*	-
C4	medium	body base C/D	2x body base C/D* 1x grey* 4x taiga*	2x body base C/D* 1x grey* 4x taiga*	2x taiga* 2x body 7*	3x terra* 2x body 7*

* Mix materials as listed. The reference quantity is the small end of the scooper.

General firing recommendation:

Pre-heating	Closing	Pre-heating time	Vacuum	Heating rate	Final temp	Holding time
450°C	2:00 min	1:00 min	off	55°C/min	800°C	1:20min

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DeguDent GmbH