Duceragold® Kiss

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Veneering with Vi-Comp LFC

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Rivestimento con Vi-Comp LFC

Recubrimiento con Vi-Comp LFC

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Duceragold Kiss has been application-tested and approved in combination with Vi-Comp LFC. Make sure to follow the Instructions for Use for Duceragold Kiss, where you will find information on contraindications, warnings, precautions and technical specifications.

Indications

Duceragold Kiss is a ceramic material suitable for veneering; metal copings and frameworks for single crowns and bridges with a coefficient of thermal expansion (CTE) of 16.2–17.6 $\mu\text{m}/\text{m} \cdot \text{K}$ (25–600°C); the non-precious alloy, Vi-Comp LFC, with a CTE of 16.1 $\mu\text{m}/\text{m} \cdot \text{K}$ (25–600°C); or the pressable framework glass ceramics, Cergo

Tips and tricks for use with Vi-Comp LFC

Framework preparation

Recommendation: sandblast – oxidize – sandblast
Sandblast with alumina (250 μm , 3–4 bar)
Oxidize at atmospheric pressure (850°C/10 min)
Sandblast with alumina (250 μm , 3–4 bar)

Veneering recommendation

Opaque firings	790°C
Dentine firings	790°C, rapid cooling
Glaze	780°C, rapid cooling

To eliminate discolouration on lighter shades, it is recommended to brush off and then steam-clean the framework under running water after each firing.

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DeguDent GmbH