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## Ducera® Lay Connectorpaste

### Ducera Lay Connectorpaste

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## Ducera Lay Connectorpaste

### Ducera Lay Connectorpaste

The main advantage of the Ducera Lay Connectorpaste is that it is easy to apply and that it can be applied more evenly than a powder mixture. In addition, working with Ducera Lay Connectorpaste is time saving as it is not necessary to constantly saturate the Ducera Lay die.

#### Indications for Use

Ducera Lay Connectorpaste is a ready-to-use porcelain paste. It is used to seal the surface of the Ducera Lay refractory material after the firing and thus ensures good bonding between the refractory material and the porcelain.

#### Contraindications

None known.

#### Warnings

May be fatal if swallowed and enters airways.

Dipose of contents / Container in accordance with local regulation.

#### Precautions

Important: Do not use any saturation liquid! Do not saturate the dies!

Ducera Lay Connectorpaste is water-repellent and should not come in contact with water (risk of streaks).

#### Adverse Reactions

None known.

#### Step-by-Step instructions

Produce Ducera Lay dies as usual.

Let the Ducera Lay die cool down after sinterfiring and ensure that the surface is dry. Then apply Ducera Lay Connectorpaste in a thin, covering layer using the attached short-hair brush. Ducera Lay Connectorpaste should be applied slightly over the preparation edge (1– 2 mm).

Use Ducera Lay cleaner to clean the brush.

#### Firing Chart:

Predrying:	1 min
Preheating:	3 mins
Preheating temperature:	620 °C
Heatrate:	55 °C under vacuum
Final temperature:	970 °C
Holding time:	1 min

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DequDent GmbH